
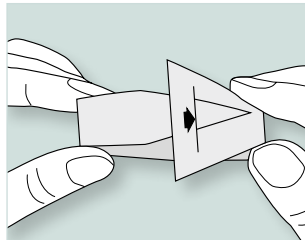


# Measuring tape for you to print out yourself and cut out in original size\*


## Measuring Tape

The measuring tape is used to determine the exact size of the required urinary sheath. The better the fit of the urinary sheath, the safer it will bond to the skin.


To measure, put the measuring tape around the middle of the penis shaft. If the urinary sheath is usually fixed to an erect penis, the penis should also be measured in this condition.

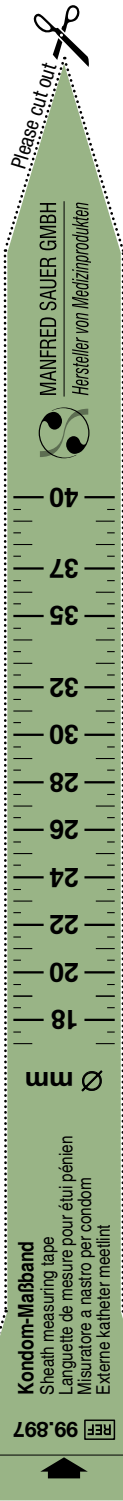

Thread the tip of the measuring tape through the slit from behind.

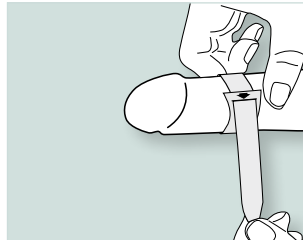

Slip the measuring tape over the penis and tighten it around the middle of the penis shaft making sure it fits firmly without being too tight.

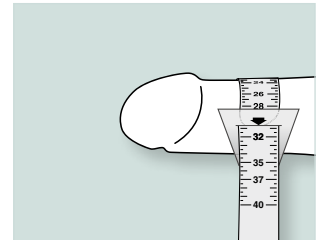

If possible, read the size with the tape still in place. The arrowhead indicates the required sheath diameter (e.g. 30 mm). If the arrowhead points at the space between two sizes, please choose the smaller size.

### \*Directions for your own printout:

1. Print this page as a DIN A4 document at 100 % scale.
2. Make sure that you do not specify any scaling under printer settings; in Adobe Reader use the "Actual size" setting for printing.
3. To make sure, you can check whether this line is 8 cm long.



If it is not 8 cm long, your printer settings are incorrect!

Please cut a slit along the breadth of the dotted line shown

Telephone counselling: +49 6226 960 104 • Mon-Fr 10.00–16.00 h, Fri 8.00–15.00 h

German Time



MANFRED SAUER GMBH  
Hersteller von Medizinprodukten

Neurott 7–15 • 74931 Lobbach • Germany  
Tel. +49 6226 960 104 • Fax +49 6226 960 011  
export@manfred-sauer.de • www.manfred-sauer.com