


Functional packaging

Variable use

The functional packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front (tip end) to seal the packaging.

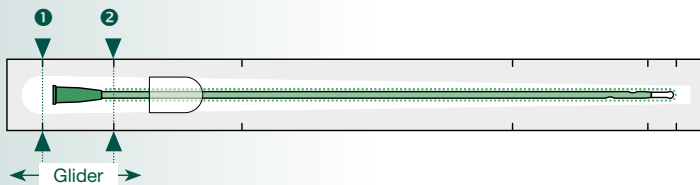


Tear-off perforations ▼ tear open here

- 1 Opening
- 2 Tearing off by jerks will create a piece of film (glider) that can be used as insertion aid

Single-use Gel Catheter

IQ 3204.xx length 40 cm, **immediately ready for use**, prelubricated with gel, functional packaging, sterile

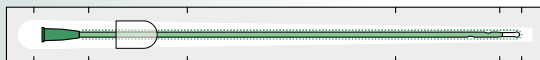

	Size*
●	10 3,3
○	12 4,0
●	14 4,7
●	16 5,3
●	18 6,0

* The size of the catheter refers to its external diameter indicated in French gauge (CH) and millimetres. The funnel is coloured to code the respective size.

Tel. +49 6226 960 104
export@manfred-sauer.de

Do you have questions regarding the single-use catheter IQ-Cath? Please give us a call or send us an e-mail.



XE 2019-11
REF 99.931



4

1250274115446941

MADE IN GERMANY

We develop and manufacture our products in Lobbach, Germany.



MANFRED SAUER GMBH
Hersteller von Medizinprodukten

Neurott 7-15 • 74931 Lobbach • Germany
Tel. +49 6226 960 104 • Fax +49 6226 960 011
export@manfred-sauer.de • www.manfred-sauer.com



MANFRED SAUER GMBH

Hersteller von Medizinprodukten

iQCATH

Immediately ready for use – Gel

Single use catheter for particularly gentle catheterisation

The catheter with 3 functional zones

To allow an easy and gentle insertion of the catheter into the urethra, the versions with a **length of 40 cm or 30 cm** are divided in three functional areas:

Rounded tip (patented)

The tip of the IQ-Cath consists of a soft, rounded head that sits on top of a highly bendable "neck". This combination allows the catheter to pass bottle-necks and bulges of the urethra nearly without resistance.

Flexible zone (patented)

The tip is followed by an approximately 5 cm long area that is flexible. It enhances the ability of the catheter to follow the run of the urethra without pressure. (this doesn't apply to the catheters with a length of 12 cm).

Smooth eyelets


The catheter eyelets are rounded to prevent any injuries to the sensitive tissue of the urethra mucosa.

Guiding zone


This part of the IQ-Cath, which continues down to the funnel, is made of a comparatively stiffer material thus making insertion easier.

4 – 6 times a day, 150 times a month, 1800 times a year.
This is about as often as an average user – whether male or female – of the intermittent self-catheterisation inserts a catheter through the urethra to empty the bladder. Here, the quality and the application safety of the catheter are crucial to prevent injuries from the beginning.


Application description – The following information is not a substitute for a visit to a doctor and careful reading of the instructions for use.

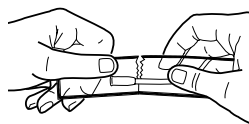


Open the aluminium bag, take one catheter and open the functional packaging at the tear-off perforation near the funnel end.

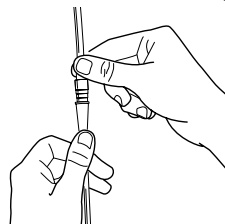


Then connect the catheter to a urine collection bag, if required.

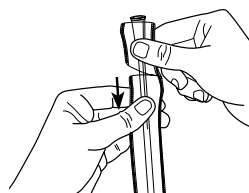

Use the next perforation to tear off the functional packaging by jerks thus obtaining a 5 cm-long piece of film (glider) that can be used as insertion aid.

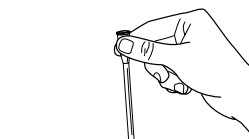

Move the glider towards the rounded tip ...

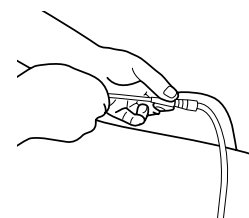

... and use it to insert the catheter safely and without touching it directly.

IQCATH 32 – USER FILMS:

User Films of the IQ-Cath created by affected persons for the benefit of other affected persons. To help you find the best way of handling the IQ-Cath in accordance with your personal requirements.



Rounded tip

Flexible zone

Guiding zone