




Application description – The following information is not a substitute for a visit to a doctor and careful reading of the instructions for use.

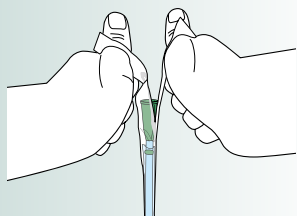

Hold the catheter vertically with the tip downwards and open the “Peel” packaging at the funnel end.

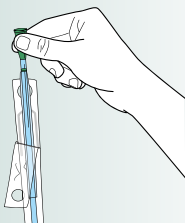


Remove the catheter from the plastic peel film holding it at the funnel and connect it to a urine collection bag, if required.

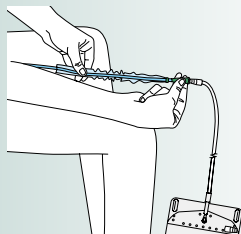

Insert the catheter until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

2 lengths

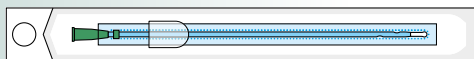

Tel. +49 6226 960 104
export@manfred-sauer.de

Do you have questions regarding the single-use catheter IQ-Cath? Please give us a call or send us an e-mail.



XE 2019-11
REF 99.910



MADE IN GERMANY

We develop and manufacture our products in Lobbach, Germany.



MANFRED SAUER GMBH
Hersteller von Medizinprodukten

Neurott 7–15 • 74931 Lobbach • Germany
Tel. +49 6226 960 104 • Fax +49 6226 960 011
export@manfred-sauer.de • www.manfred-sauer.com



MANFRED SAUER GMBH

Hersteller von Medizinprodukten

iQ CATH

The catheter with 3 functional zones with “Sleeve”



Single use catheter for particularly gentle catheterisation

The catheter with 3 functional zones

To allow an easy and gentle insertion of the catheter into the urethra, the versions with a **length of 40 cm or 30 cm** are divided in three functional areas:

Rounded tip (patented)

The tip of the IQ-Cath consists of a soft, rounded head that sits on top of a highly bendable "neck". This combination allows the catheter to pass bottle-necks and bulges of the urethra nearly without resistance.

Flexible zone (patented)

The tip is followed by an approximately 5 cm long area that is flexible. It enhances the ability of the catheter to follow the run of the urethra without pressure. (this doesn't apply to the catheters with a length of 12 cm).

Smooth eyelets

The catheter eyelets are rounded to prevent any injuries to the sensitive tissue of the urethra mucosa.

Guiding zone

This part of the IQ-Cath, which continues down to the funnel, is made of a comparatively stiffer material thus making insertion easier.

IQ-CATH-USER FILMS:

User Films of the IQ-Cath created by affected persons for the benefit of other affected persons. To help you find the best way of handling the IQ-Cath in accordance with your personal requirements.



4 – 6 times a day, 150 times a month, 1800 times a year.
This is about as often as an average user – whether male or female – of the intermittent self-catheterisation inserts a catheter through the urethra to empty the bladder. Here, the quality and the application safety of the catheter are crucial to prevent injuries from the beginning.

Sleeve

No-touch insertion

The sleeve, a thin film, covers the catheter completely and permits a no-touch insertion.



Peel packaging


Easy handling

Finger holes allow to pull open ("peel") the packaging easily without touching the catheter itself.




Versions of the IQ Cath with Sleeve (insertion aid) Hydrophilic Single-use Catheter with Sleeve

IQ 4304.xx length 40 cm, **immediately ready for use**, sterile saline solution 0.9 %, activated, Sleeve (insertion aid), "Peel" packaging, sterile

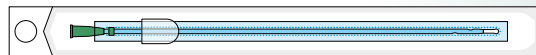

IQ 4303.xx length 40 cm, **immediately ready for use**, sterile saline solution 0.9 %, activated, Sleeve (insertion aid), "Peel" packaging, sterile

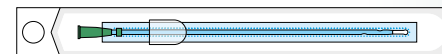

Size*
CH mm

- 10 3,3
- 12 4,0
- 14 4,7
- 16 5,3
- 18 6,0

- 10 3,3
- 12 4,0
- 14 4,7

* The size of the catheter refers to its external diameter indicated in French gauge (CH) and millimetres. The funnel is coloured to code the respective size.

Rounded tip

Flexible zone

Guiding zone