

Versions of the IQ-Cath

Length of the catheter

The length specification refers to the length of the catheter without the funnel: 40cm mainly for men, 30cm mainly for boys, 12cm mainly for women. The lengths mentioned below are not available for all catheter versions.

Size	●	○	●	●	●
CH	10	12	14	16	18
mm	3,3	4,0	4,7	5,3	6,0

The size of the catheter refers to its external diameter indicated in French gauge (CH) and millimetres. The funnel is coloured to code the respective size.

Hydrophilic Single-use Catheter

	IQ-Cath	Ready for Use	Extra	Length	Size in CH
Functional Packaging	20 <input type="checkbox"/>	No	–	40 cm	● ○ ● ● ● X
	21 <input type="checkbox"/>	Yes	–	40 cm	● ○ ● ● ● ●
“Peel” Packaging	29	No	–	40 cm	● ○ ● ● ● ●
	29	No	–	12 cm	● ○ ● ● X X
	23	Immediately	–	40 cm	● ○ ● ● ● ●
	23	Immediately	–	30 cm	● ○ ● ● X X
	23	Immediately	–	12 cm	● ○ ● ● X X
	43	Immediately	Sleeve	40 cm	● ○ ● ● ● ●
43	Immediately	Sleeve	30 cm	● ○ ● ● X X	
Urine Col-lection Bag	26 <input type="checkbox"/>	Yes	–	40 cm	X ○ ● ● ● ●
				<input type="checkbox"/> Partly also available with additional wrapping	10 12 14 16 18

Single-use Gel Catheter

	IQ-Cath	Ready for Use	Extra	Length	Size in CH
Functional Packaging	30	No	–	40 cm	X ○ ● ● ● X
	31 <input type="checkbox"/>	Yes	–	40 cm	● ○ ● ● ● ●
	32	Immediately	–	40 cm	● ○ ● ● ● ●
Urine Col-lection Bag	36 <input type="checkbox"/>	Yes	–	40 cm	X ○ ● ● ● ●
				<input type="checkbox"/> Partly also available with additional wrapping	10 12 14 16 18

Tel. +49 6226 960 104
export@manfred-sauer.de

Do you have questions regarding the single-use catheter IQ-Cath? Please give us a call or send us an e-mail.



MADE IN GERMANY We develop and manufacture our products in Lobbach, Germany.

MANFRED SAUER GMBH Hersteller von Medizinprodukten
Neurott 7–15 • 74931 Lobbach • Germany
Tel. +49 6226 960 104 • Fax +49 6226 960 011
export@manfred-sauer.de • www.manfred-sauer.com




MANFRED SAUER GMBH

Hersteller von Medizinprodukten

iQCATH

The catheter with 3 functional zones

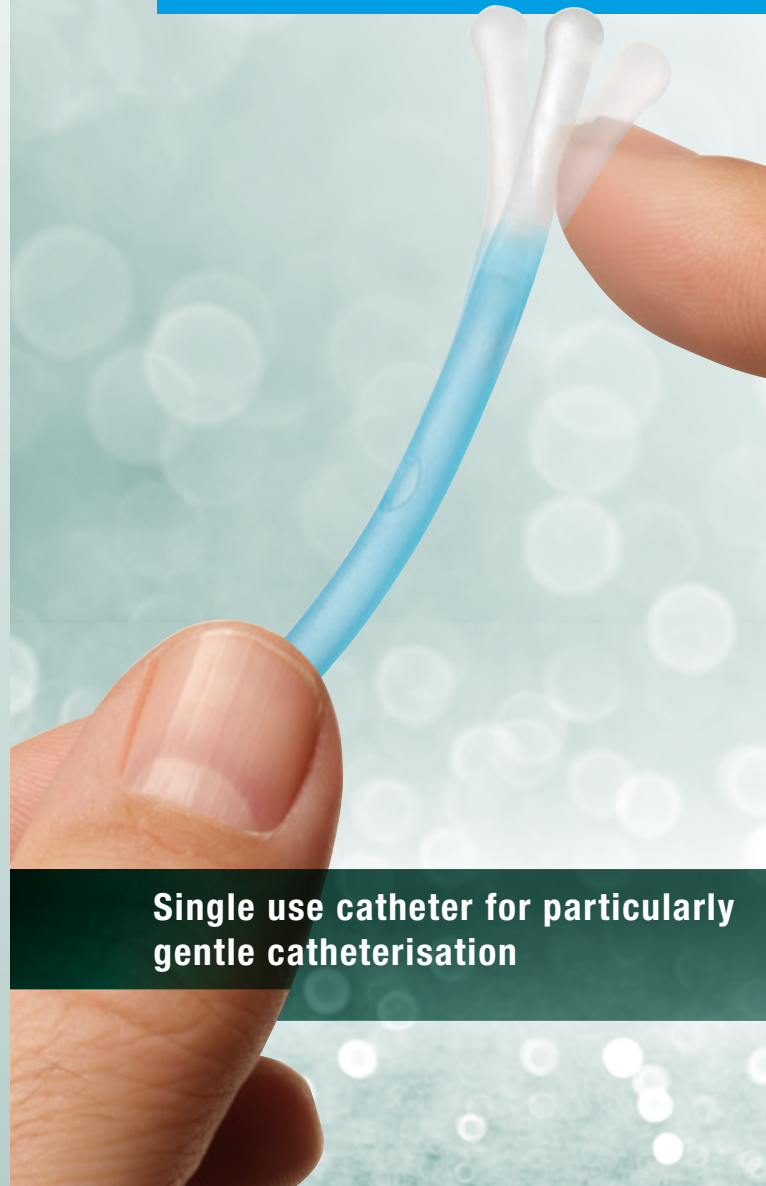

Single use catheter for particularly gentle catheterisation

The catheter with 3 functional zones

To allow an easy and gentle insertion of the catheter into the urethra, the versions with a **length of 40 cm or 30 cm** are divided in three functional areas:

Rounded tip (patented)

The tip of the IQ-Cath consists of a soft, rounded head that sits on top of a highly bendable "neck". This combination allows the catheter to pass bottle-necks and bulges of the urethra nearly without resistance.

Flexible zone (patented)

The tip is followed by an approximately 5 cm long area that is flexible. It enhances the ability of the catheter to follow the run of the urethra without pressure. (this doesn't apply to the catheters with a length of 12 cm)

Smooth eyelets

The catheter eyelets are rounded to prevent any injuries to the sensitive tissue of the urethra mucosa.

Guiding zone

This part of the IQ-Cath, which continues down to the funnel, is made of a comparatively stiffer material thus making insertion easier.

IQ-CATH-USER FILMS:

User Films of the IQ-Cath created by affected persons for the benefit of other affected persons. To help you find the best way of handling the IQ-Cath in accordance with your personal requirements.



4 – 6 times a day, 150 times a month, 1800 times a year.
This is about as often as an average user – whether male or female – of the intermittent self-catheterisation inserts a catheter through the urethra to empty the bladder. Here, the quality and the application safety of the catheter are crucial to prevent injuries from the beginning.

Sleeve

No-touch insertion


The sleeve, a thin film, covers the catheter completely and permits a no-touch insertion.



Peel packaging

Easy handling


Finger holes allow to pull open ("peel") the packaging easily without touching the catheter itself.

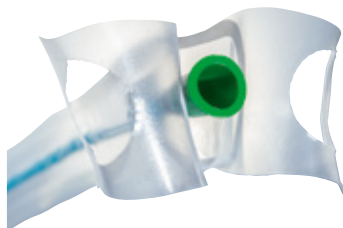

Tear-off perforations

▼ tear open here

- 1 Opening
- 2 Tearing off by jerks will create a piece of film (glider) that can be used as insertion aid

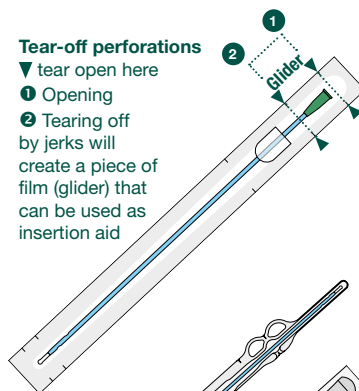


Functional packaging

Variable use

The functional packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front (tip end) to seal the packaging.

All In One Version...

... for the hydrophilic catheter or for the catheter for use with gel. Capacity of the collection bag 1 litre.

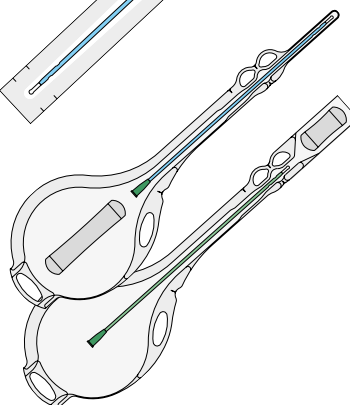

Rounded tip

Flexible zone

Guiding zone