

Aids for ISC

The ISC-aids were developed in close co-operation with male and female users (paraplegics and quadriplegics) and occupational therapists. Our aim was to ensure and facilitate an independent and safe ISC (Intermittent Self-Catheterization) procedure.



MANFRED SAUER GMBH

Hersteller von Medizinprodukten

iQ CATH

... for women



Leg spreader

REF 110.02

Mirror with Velcro

REF 110.33



Draining Set

REF 110.65

The tube that can be used as an extension for the catheter to drain the urine safely into the toilet.

Tel. +49 6226 960 104
export@manfred-sauer.de

Do you have questions regarding the single-use catheter IQ-Cath? Please give us a call or send us an e-mail.



XE 2020-10
REF 99.913



MADE IN GERMANY We develop and manufacture our products in Lobbach, Germany.



MANFRED SAUER GMBH
Hersteller von Medizinprodukten

Neurott 7-15 • 74931 Lobbach • Germany
Tel. +49 6226 960 104 • Fax +49 6226 960 011
export@manfred-sauer.de • www.manfred-sauer.com

Single use catheter for particularly gentle catheterisation

4 – 6 times a day, 150 times a month, 1800 times a year.
 This is about as often as an average user – whether male or female – of the intermittent self-catheterisation inserts a catheter through the urethra to empty the bladder. Here, the quality and the application safety of the catheter are crucial to prevent injuries from the beginning.

Composition

Rounded tip (patented)

The tip of the IQ-Cath consists of a soft, rounded head that sits on top of a highly bendable “neck”. This combination allows the catheter to pass bottlenecks and bulges of the urethra nearly without resistance.

No flexible zone

In contrast to the 3-zone catheter, the IQ-Cath for women or girls has no flexible zone.



Smooth eyelets:

The catheter eyelets are rounded to prevent any injuries to the sensitive tissue of the urethra mucosa.

Guiding zone

This part of the IQ-Cath, which continues down to the funnel, is made of a comparatively stiffer material thus making insertion easier.


Connector

The extra-long connector allows for a safe grip and better handling properties.



Peel packaging Easy handling

Finger holes allow to pull open (“peel”) the packaging easily without touching the catheter itself.

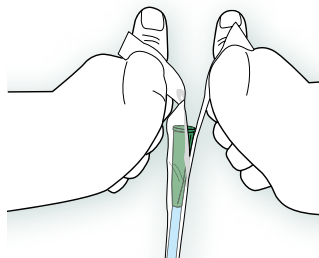

Hold the catheter vertically with the tip downwards and open the “Peel” packaging at the funnel end. Avoid touching any part of the catheter surface (except for the funnel) directly.

Length of the catheter


The length specification refers to the length of the catheter without the funnel.

Hydrophilic Single-use Catheter

IQ 2902.xx length 12 cm, not ready for use, “Peel” packaging, sterile



IQ 2302.xx length 12 cm, immediately ready for use, saline solution 0.9 % activated, “Peel” packaging, sterile

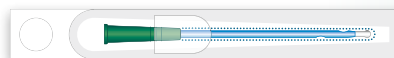

Size*

CH mm

● 10 3,3

○ 12 4,0

● 14 4,7

● 10 3,3

○ 12 4,0

● 14 4,7

*The size of the catheter refers to its external diameter indicated in French gauge (CH) and millimetres. The funnel is coloured to code the respective size.

Rounded tip

Guiding zone

Long connector