

# SAUER-Comfort

## – self-adhesive urinary sheaths


The sheath and the adhesive are made of synthetic materials. The sheath and the connecting tube are manufactured as one piece. They can be connected to all urine collection bags with or without adapter. Available in 9 sizes.

### SAUER-Comfort – Violet

Length 75 mm, buffer zone 45 mm,  
adhesive zone 25 mm, strong adhesive



Ø mm	Order no.
18	97.4511.18
20	97.4511.20
22	97.4511.22
24	97.4511.24
26	97.4511.26
28	97.4511.28
30	97.4511.30
32	97.4511.32
35	97.4511.35

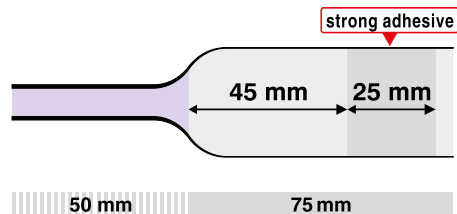

Do you have questions regarding self-adhesive sheaths?  
Would you like us to supply you with free samples?

Please give us a call or  
send us an e-mail:

Tel. +49 6226 960 104  
export@manfred-sauer.de



**MADE IN GERMANY** We develop and manufacture our products at Lobbach, Germany

 **MANFRED SAUER GMBH** Neurt 7–15 • 74931 Lobbach • Germany  
Tel. +49 6226 960 104 • Fax +49 6226 960 011  
export@manfred-sauer.de • www.manfred-sauer.com  
*Hersteller von Medizinprodukten*



**MANFRED SAUER GMBH**  
*Hersteller von Medizinprodukten*

## SAUER-Comfort

... security and simple handling

Sauer  
*Comfort*  
violet




The self-adhesive urinary sheath  
with the short adhesive zone

30 pieces in one box, 1 protective cloth (matching the size of the sheath) is enclosed.

## Problems


When using a urinary sheath, various problems may occur, which often also originate from the properties of the urinary sheath in question.

To counteract the problems listed below, a urinary sheath has to fulfil particular requirements.

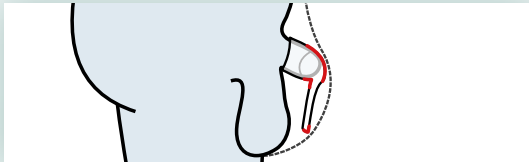


### Friction

Urinary sheaths without a flexible buffer zone may rub against the clothing, which may cause the urinary sheath to come loose untimely.

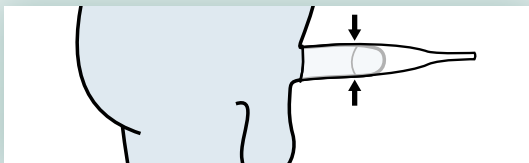


### Not the right size

Urinary sheaths that are not the right size usually don't allow for a safe adhesion to the skin.

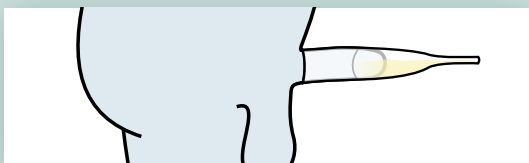

### Formation of urine residues

With urinary sheaths made of a more rigid material often a residue of urine remains in the sheath.

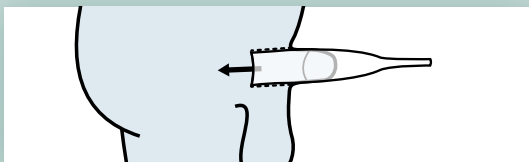

### Retracted penis

The penis withdraws in the abdominal cavity leaving only a small surface for adhesion.

## SAUER-Comfort – Violet

These urinary sheaths have been designed and are manufactured in our factory at Lobbach. They have the necessary characteristics to ensure a secure, reliable and simple handling in addition to a high wearing comfort. With the urinary sheath SAUER-Comfort – Violet the adhesive zone is significantly shorter than with our other sheaths. This feature makes them particularly suitable for users with a smaller anatomy, but without large amounts of spontaneous urine. Because of its minor length, the smaller sizes of this sheath are also suitable for children.

### ❶ Connecting tube

To connect to all urine collection bags with and without an adapter.

Length of the connecting tube: 50 mm

### ❷ Buffer zone

The sheath can be joined more closely to the body. This way, friction between the underwear and the sheath, which might otherwise cause the sheath to come off, is largely avoided. The tube between the sheath and the urine collection bag can be placed to the left or to the right side at will.

Length of the buffer zone: 45 mm

### ❸ Short adhesive zone

In case of a small surface available for adhesion, a short adhesive zone is an advantage, because no sticky ring remains at the upper end of the sheath, which might otherwise get stuck to the underwear. In compensation, the adhesive is stronger and ensures a safe adhesion. Within the adhesive zone, the urinary sheath is transparent making an assessment of the skin possible at any time without having to remove the sheath.


Length of the adhesive zone: 25 mm

### ❹ 9 sizes

The finely graded sizes (diameter 18–35 mm) allow an exact fit. The better the fit, the safer the adhesive contact.

### Measuring Tape

To determine the exact size. Free of charge.

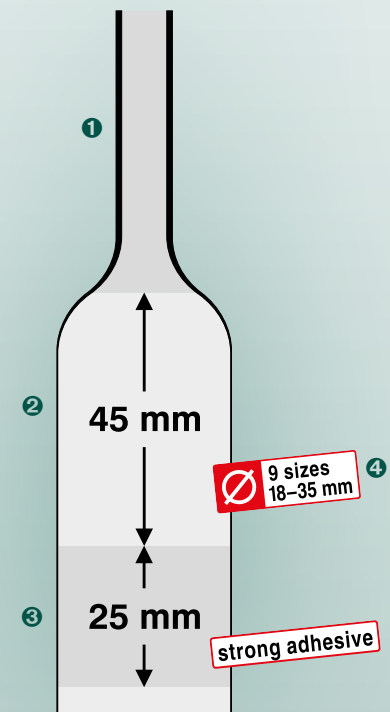

### 100 % tested

To guarantee maximum safety all urinary sheaths are tested *individually* for impermeability and tear strength.