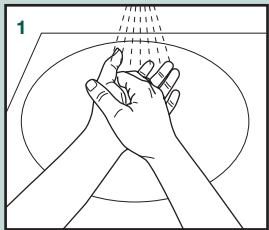
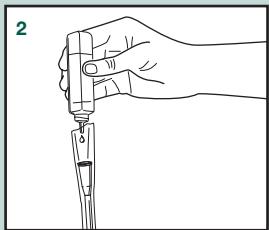
Deutsch

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English

Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly. You can use the functional packaging as gripping aid.

Open the functional packaging using the perforation line nearest to the funnel end. Hold the catheter vertically with the tip down and fill in a sufficient quantity of sterile saline solution 0.9% to ensure that all of the catheter surface (except for the funnel) is moistened for at least 30 sec.

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Turn the packet upside down and drain the fluid out. Use tear marks to open the functional packaging at the front end and then tear off with a jerk to release the tip of the catheter. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

The packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front to seal the packet.

Alternatively, you can hold the catheter at the funnel and remove the packaging completely. Attach a urine bag to the funnel if necessary.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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