Deutsch

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English

Important!
Aluminium pouches containing 10 catheters each provide an additional layer of protection against drying up. Condensation may form in these pouches occasionally, which does not affect the quality of the product. Use the catheters within 20 days of opening the aluminium pouch.

Open the aluminium bag and take out one catheter. Please make sure that the gel is covering the complete surface. If necessary, put the catheter on a flat surface and spread out the gel with two fingers in the packaging.

Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly. You can use the functional packaging as gripping aid.

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Use tear marks to open the functional packaging at the front end and then tear off with a jerk to release the tip of the catheter. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

The packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front to seal the packet.

Alternatively, you can hold the catheter at the funnel and remove the packaging completely. Attach a urine bag to the funnel if necessary.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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