Anwendergruppe: IQ36x4.xx – Männer

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<div> <div> </div> <div></div> </div>
<div> <div> </div> <div></div> </div>
<div> <div> </div> <div></div> </div>
Group of Users: IQ36x4.xx – Men
Intended use: <p>The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).</p> <p>Indication: The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual’s situation. The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.</p> <p>Contraindication: The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.</p> <p>Risks and Cautions: The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs. Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel. Do not use if the sterile packaging or the catheter is damaged. We recommend an operating temperature range of 15–25 °C. Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user’s country of residence. Manfred Sauer GmbH is not liable for damage or injury resulting from improper handling contrary to these instructions for use or failure to adhere to hygiene standards or relevant standards of care.</p>

<div> <div> </div> <div></div> </div>
<div> <div> </div> <div></div> </div>
<div> <div> </div> <div></div> </div>
Group of Users: IQ36x4.xx – Men
Intended use: <p>The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).</p> <p>Indication: The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual’s situation. The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.</p> <p>Contraindication: The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.</p> <p>Risks and Cautions: The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs. Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel. Do not use if the sterile packaging or the catheter is damaged. We recommend an operating temperature range of 15–25 °C. Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user’s country of residence. Manfred Sauer GmbH is not liable for damage or injury resulting from improper handling contrary to these instructions for use or failure to adhere to hygiene standards or relevant standards of care.</p>

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English

Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly. You can use the functional packaging as gripping aid.

Open the gel sachet by folding and squeezing. Make sure the whole content of the sachet is emptied into the packaging.

Spread the gel along the full length of the catheter (except for the funnel).

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Open the urine bag at the front end and release the tip of the catheter. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Push back the catheter into the urine bag and seal this by making a knot. To empty the bag tear at the back end.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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