Deutsch

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English

**Important!**  
Aluminium pouches containing 10 catheters each provide an additional layer of protection against drying up. Condensation may form in these pouches occasionally, which does not affect the quality of the product.  
Use the catheters within 30 days of opening the aluminium pouch.

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Open the aluminium bag and take one catheter.

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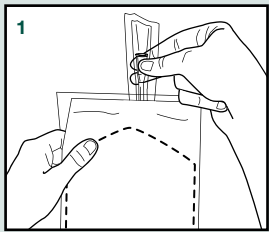
Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly.

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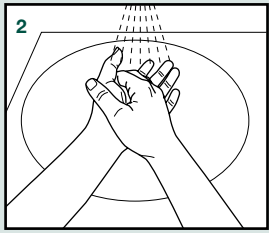
Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

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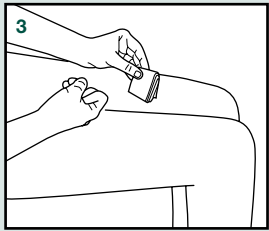
Hold the catheter vertically with the tip downwards and open the "Peel" (easy-peel packaging made of plastic film) packaging at the funnel end. Avoid touching any part of the catheter surface (except for the funnel) directly.

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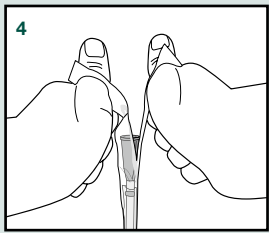
Remove the catheter from the plastic peel film holding it at the funnel. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

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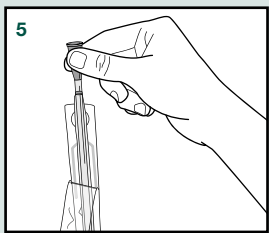
If necessary, attach a urine bag to the catheter.

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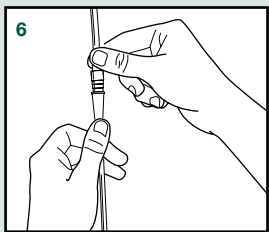
Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

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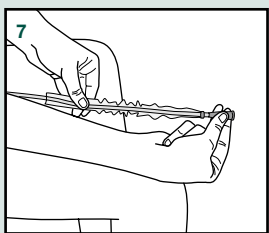
Dispose of the catheter according to local authority guidelines.

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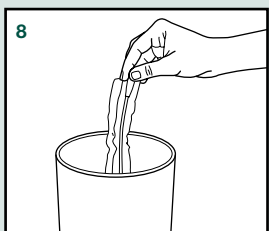
Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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