Česky

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Please wash your hands thoroughly before catheterisation.
At all times avoid touching the catheter itself. It is possible to use part of the packaging as a gripping aid.

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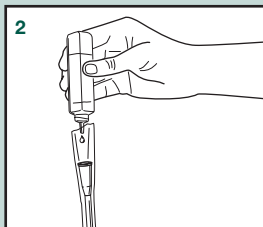
Open the packet at the funnel end. Hold catheter upright and fill with saline or clean water to ensure the catheter surface is activated by the fluid for 30 sec.

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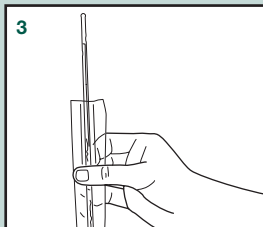
Turn the packet upside down and drain the fluid out. Open the packaging at the front end and release the tip of the catheter.

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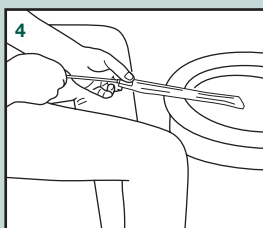
The packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front to seal the packet.

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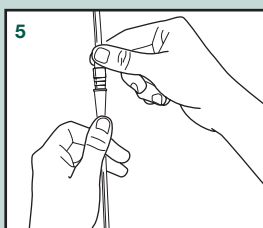
Alternatively the catheter can be removed at the funnel end. Attach a urine bag if necessary.

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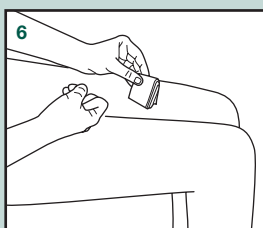
Clean the glans (tip of penis) as directed by a healthcare professional.

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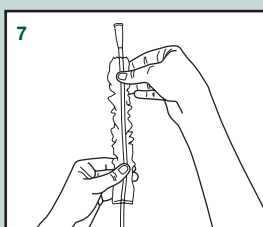
Insert the catheter until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

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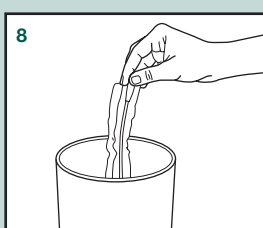
Dispose of the catheter according to local authority guidelines.

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For further information regarding our catheter systems please contact our customer advisors.

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