Deutsch

Wichtig!

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English

Important!

The catheters are packed in an aluminium bag containing 10 pcs to avoid them drying out. Once you have opened the aluminium bag the product should be used within 10 days.

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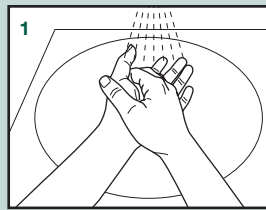
Please wash your hands thoroughly before catheterisation. At all times avoid touching the catheter itself. It is possible to use part of the packaging as a gripping aid.

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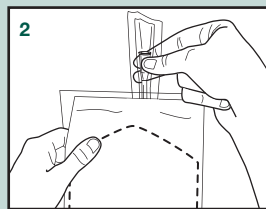
Open the aluminium bag and take one catheter. Please make sure that the gel is covering the complete surface. If necessary put the catheter on a flat surface and distribute the gel with two fingers in the packaging.

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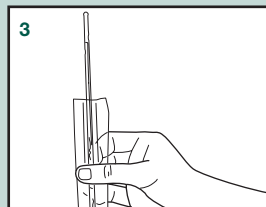
Open the packaging at the tip end and release the tip of the catheter. In all stages of catheterisation you can hold the catheter with the packaging but please avoid touching the unprotected surface of the catheter directly.

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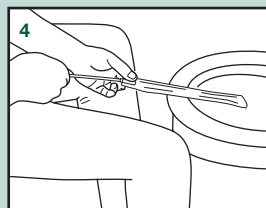
The packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front (tip end) to seal the packaging.

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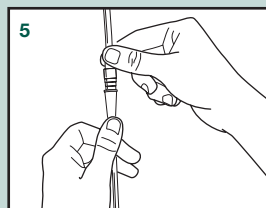
As an alternative you can hold the catheter at the funnel end and remove the packaging completely. Attach a urine bag to the funnel if necessary.

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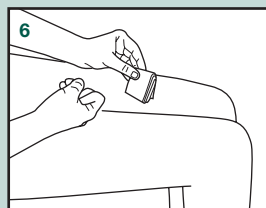
Clean the glans (tip of penis) as directed by a healthcare professional.

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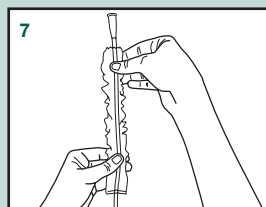
Insert the catheter until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

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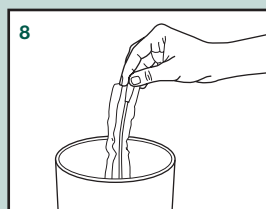
Dispose of the catheter according to local authority guidelines.

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For further information regarding our catheter systems please contact our customer advisors.

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