Deutsch

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English

**Important!**  
The catheters are packed in an aluminium bag containing 10 pcs to avoid them drying out. Once you have opened the aluminium bag the product should be used within 20 days.

Open the aluminium bag and take one catheter.

Before use, wash hands thoroughly and disinfect them.

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Hold the catheter vertically with the tip downwards and open the "Peel" (easy-peel packaging made of plastic film) packaging at the funnel end. Avoid touching any part of the catheter surface (except for the funnel) directly.

Remove the catheter from the plastic peel film holding it at the funnel.

If necessary, attach a urine bag to the catheter.

Insert the catheter until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems please turn to a medical professional or to our customer adviser.

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