


Choosing the adequate self-adhesive urinary sheath

For the care of a moderate or a severe bladder incontinence

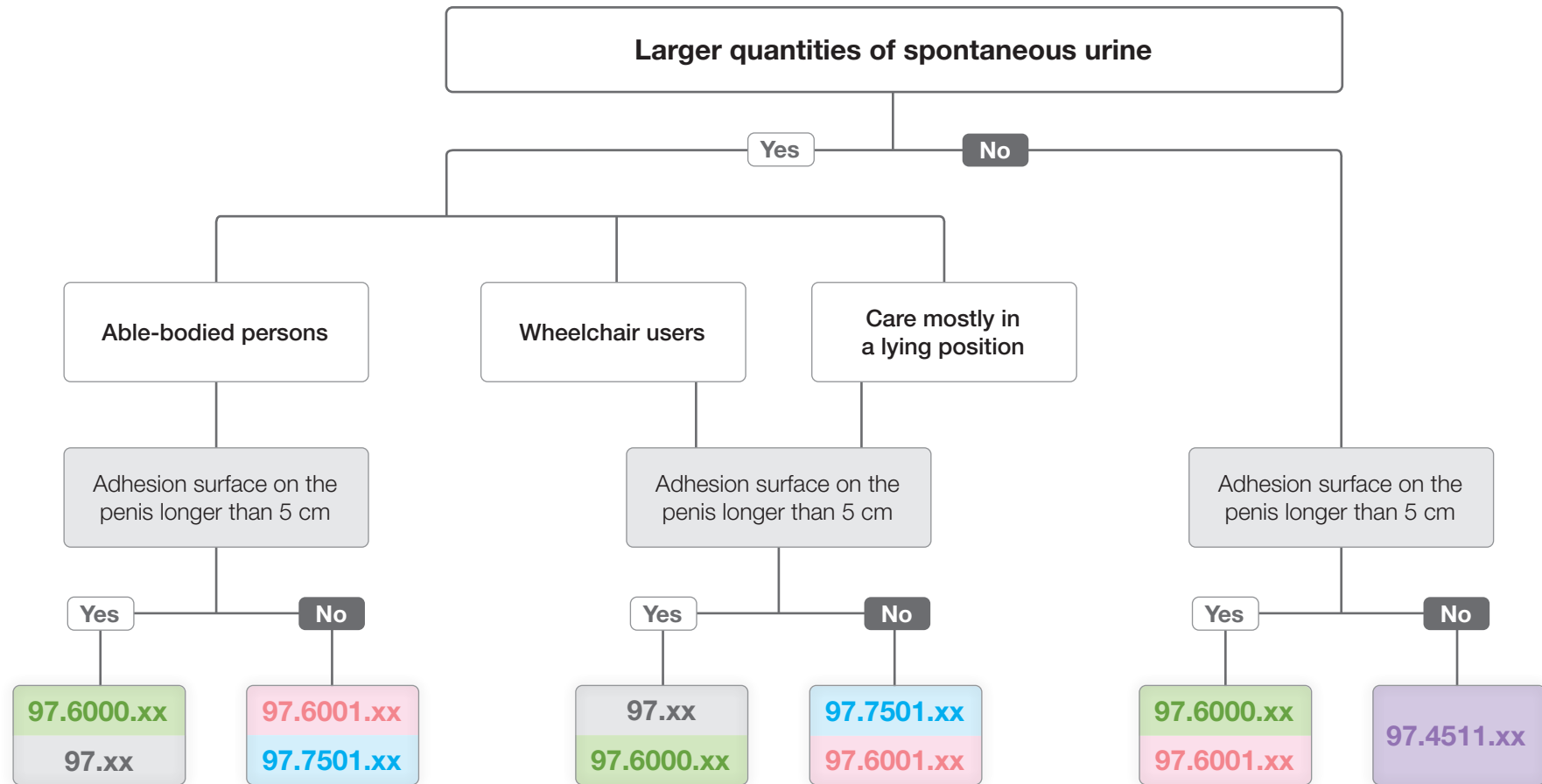

Important!

- Use appropriate sheath size (determine size using the measuring tape)
- The skin in the adhesive area should be dry and free from grease
- Choose the position of the sheath in a way to make sure that the buffer zone has the required length
- Connect sheath to the urine collection bag and make sure that the urine can drain unimpeded

Helpline: +49 6226 960 104

If the tested sheaths don't provide a satisfying result, please contact our customer service. Find out more in our "Tutorial – Sheath Drainage System", which we will send you free of charge or which you can download from our homepage.

SAUER-Comfort – self-adhesive urinary sheath

Order no.	Description	Buffer zone	Adhesive zone	Adhesive	Size
97.xx	Standard	75 mm	50 mm	standard	11 (18–40 mm)
97.7501.xx	Blue	75 mm	50 mm	strong	11 (18–40 mm)
97.6000.xx	Green	60 mm	50 mm	standard	11 (18–40 mm)
97.6001.xx	Red	60 mm	50 mm	strong	11 (18–40 mm)
97.4511.xx	Violet	45 mm	25 mm	strong	9 (18–35 mm)

In the order no. "xx" is replaced by the required size (e.g. 97.7501.26).

