Selbstklebendes Kondom-Urinal	Deutsch	2
Self-Adhesive Urinary Sheath	English	4
Étui pénien autocollant	Français	6
Condom urinario autoadesivo	Italiano	8
Zelfklevende externe katheters	Nederlands	10
Självhäftande urindroppssamlare	Svenska	12
Selvklebende urinalkondom	Dansk	14
Itseliimautuva urinaalikondomi	Suomi	16
Selvimende uridom	Norsk	18
Notkunarleiðbeiningar	Ísland	20
Isekleepuv uriinikogur	Eesti	22
Colector de orina peneano autoadhesivo	Español	24
Preservativo urinário autoadesivo	Português	26
Αυτοκόλλητος περιπεϊκός καθετήρας	Ελληνική	28
Öntapadó vizeletkondom	Magyar	30
Samoprzylepny cewnik zewnętrzny	Polski	32
Samolepicí urinální kondom	Čeština	34
Prezervativ urinar autoadeziv	Română	36
Самозалепващ уринален кондом	Български	38
Samolepilni urinal kondom	Slovenščina	40
Samoljepivi urinarni kondom	Hrvatski	42
Samolepljivi urinarni kondom	Srpski	44
Samoljepljivi urinarni kondom	Bosanski	46
Самолеплив уринарен кондом	Македонски	48
51	فارسی	کاندوم ادراری چسب دار
53	العربية	قسطرة التبول الخارجی ذاتیة الالتصاق

URINAL-KONDOM – ANTI-ALLERGEN

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Inhalt

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For Single Use

Contents

- **30 urinary sheaths**, for separate fixation, anti-allergic, made of synthetic material (without addition of latex), powdered, manufactured as one piece with the connecting tube, connects to all leg bags and bed bags with and without adapter
- **Length 150 mm**, buffer zone: individual, fixation with skin adhesive or double-sided adhesive tape
- **8 sizes:** ø 24, 26, 28, 30, 32, 35, 40, 44 mm
("xx" in the order no. represents the size of the sheath, e.g. 56.30 = 30 mm)
- **1 protective cloth**, matching the size of the sheath

Intended use

SAUER urinary sheaths are used for external urinary drainage into a urine collection bag in men with urinary incontinence or if external urinary drainage is desired for other reasons. They are used on intact skin.

Indications

Urinary incontinence of varying causes without residual urine (e.g., neurogenic urinary dysfunctions, hyperactive bladder, urinary incontinence after surgery); urinary drainage in people who cannot manage to go to the toilet by themselves.

Contraindications

Chronic urinary retention with high intravesical pressure; sphincter dyssynergia with significant urinary retention; constrictions in the urethra (e.g., prostatic hyperplasia) resulting in high residual amounts of urine; skin abrasions on the penis (e.g., wounds, eczema, pressure sores); acute skin diseases (e.g., mycosis, bacterial skin infections); allergies / incompatibilities to the material or adhesive/adhesive spray/adhesive tape; pronounced phimosis.

Important information on hygiene and period of wear

When using continence products, the hygiene guidelines of the national health authorities must be observed. We recommend changing the sheath (the adhesive tape, bag) after 24 hours. The period of wear may also be less than 24 hours depending on the skin type.

Selecting the right size/product

SAUER Urinary Sheaths – Anti-Allergic are available in 8 sizes. It is essential to select the right size so that the sheath fits properly and does not cause leaks or pressure sores. To select the size, the circumference of the penis must be measured around the middle of the shaft. If it is adhered during erection (see application), the circumference is also measured on the erect penis. The product-specific measuring tapes state the diameter based on the circumference measurement. If leaks or pressure sores occur, the sheath size must be checked and adjusted if necessary.

Risks and warnings

Ask your doctor if urinary sheaths are suitable for your incontinence care. Instructions on how to use urinary sheaths should be provided by a qualified nurse.

The use of urinary sheaths may cause skin irritation and skin changes. When removing a urinary sheath, inspect the skin. If you notice any abnormalities, stop using the product and seek medical advice.

If urinary sheaths are worn too tightly or if the adhesive tape is applied too tightly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constrictions and necrosis. In this case, seek a doctor without delay. When using adhesive tapes, the risk of constrictions/necrosis is greater than when using self-adhesive sheaths or adhesive sprays. When applying the adhesive tape, the application instructions must therefore be followed exactly.

APPLICATION

Preparations

Wash the skin in the whole genital area thoroughly and dry well. If used on damp skin or on skin that was pre-treated with oily/greasy products, the adhesive effect is reduced.

An erection is not absolutely necessary for adhesion, but if one can be manipulated, we recommend that you apply the sheath to an erect penis as this gives a more stable surface for adhesion and prevents constriction.

Optional: Pull over the protective cloth so that the pubic hair does not stick or obstruct adhesion. The urinary sheath can be attached to the penis with medical adhesive tape, adhesive spray or skin adhesive suitable for the skin. Please follow the manufacturer's instructions.

Individually adjusting the buffer zone

The length of the buffer zone, which is used to collect spontaneous urine, can be individually set. To do this, unroll the sheath to the desired length of the buffer zone before fixing it to the shaft.

Fixing with skin adhesive / adhesive spray

Unroll the buffer zone as far as desired and position the sheath on the foreskin without smoothing the foreskin back. If there is no foreskin, place the urinary sheaths directly on the glans (Fig. 1).

With the other hand, apply a ring of adhesive to the penis or spray the adhesive spray on the penis. (Fig. 2). You may have to wait a little for the solvent in the adhesive to evaporate, as this could impair the sheath.


Use one hand to unroll the sheath over the adhesive on the shaft, while using the other hand to keep the penis taut (Fig. 3). Press the sheath carefully around the penis and, if necessary, rub out longitudinal folds and air bubbles.


Apply the sheath


Apply the adhesive/spray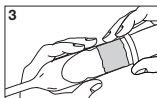

Roll the sheath over the adhesive area

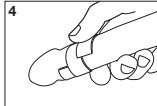Apply the adhesive tape spirally so that it overlaps

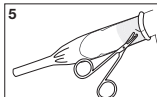Remove the ring

Fixing with adhesive tape

Apply the adhesive tape in a spiral so that it overlaps and is not too tight on the penis (Fig. 4). Do not stretch the tape. Then, as described above, unroll the buffer zone as far as desired and position the sheath on the foreskin without smoothing the foreskin back. Slightly stretch the connecting tube. Use one hand to unroll the sheath over the adhesive band on the shaft, while using the other hand to keep the penis taut. Press the sheath carefully around the penis.

Removing the ring

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 5). Do not use scissors with pointed tips as they may cause injury. The incision should be circular in shape so that the sheath does not tear. When cutting or tearing off, press down on the adhesive area of the sheath at the same time.

Connecting to the bag

Push the connecting tube of the sheath on the adapter of the bag and press all around gently. For bags without adap-

ter, push the connecting tube directly on the inlet tube. Make sure that the buffer zone and inlet tube are not twisted or bent.

Removing the sheath

Remove the sheath after 24 hours at the latest and pull or roll off from the back towards the tip of the penis. Do not pull too strongly so as to avoid skin abrasions. An oil pad can be used to undo the adhesive bond. Carefully remove any adhesive remains.

Disposal

Dispose of the sheath in household waste.

	Catalog number		Observe instructions for use
	Store in a dry place		CE marked medical device
	Quantity		Use by (year-month-day)
	Keep away from sunlight		Not for reuse
	Swiss authorised representative		Pharmaceutical central no.
	UK authorised representative		Drug Tariff
	Batch number		List of products and services
			Manufacturer

If serious incidents occur when using urinary sheaths – Anti-Allergic, these must be reported to the manufacturer and competent authority of the country in which the user resides.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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