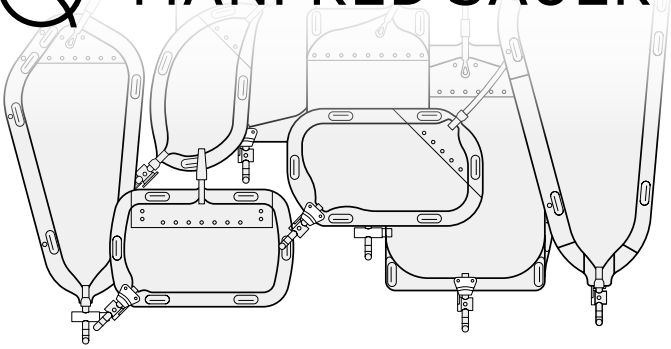
MANFRED SAUER

SAUER-Urinbeutel

STERILE

Abbildungen/Figures/Illustraties/Kuva/Illusztrációk/قِي حِي ضوئال موصولل

2

Gebrauchsanweisung	Deutsch	3
Instructions for use	English	5
Notice d'utilisation	Français	7
Istruzioni per l'uso	Italiano	9
Gebruiksaanwijzing	Nederlands	11
Bruksanvisning	Svenska	13
Brugsanvisning	Dansk	15
Käyttöohje	Suomi	17
Bruksanvisning	Norsk	19
Notkunarleiðbeiningar	Ísland	21
Kasutusjuhised	Eesti	23
Instrucciones de uso	Español	25
Instruções de utilização	Português	27
Οδηγίες χρήσης	Ελληνική	29
Használati utasítás	Magyar	31
Instrukcja użytkowania	Polski	33
Návod k použití	Čeština	35
Instrucțiuni de utilizare	Română	37
Инструкция за употреба	Български	39
Navodila za uporabo	Slovenščina	41
Upute za uporabu	Hrvatski	43
Uputstvo za upotrebu	Српски/srpski	45
Uputstvo za upotrebu	Bosanski	47
Упатство за употреба	Македонски	49
دستور العمل استفاده	فارسی	51
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	53









MANFRED SAUER GmbH

Neurott 7-15 · 74931 Lobbach · Germany
Tel. +49 6226 960 100 · info@manfred-sauer.de
www.manfred-sauer.com

20260216 \$22765

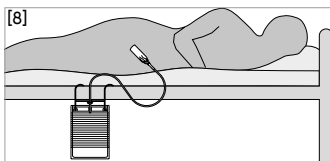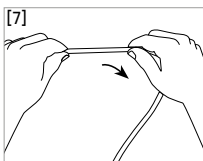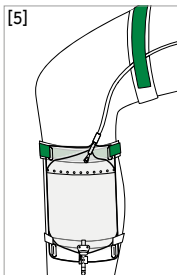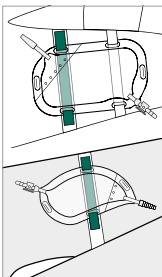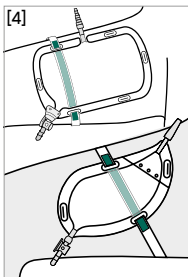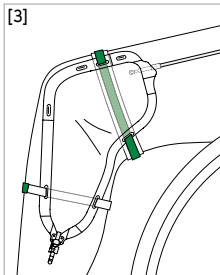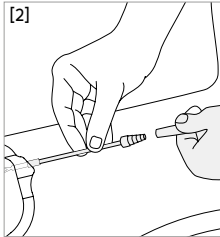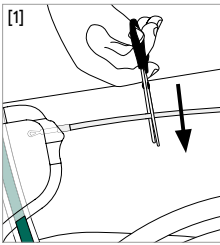CE 0123

Deutsch

Sterile SAUER-Urinbeutel

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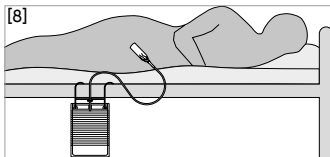
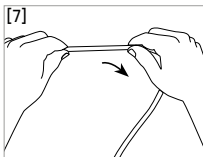
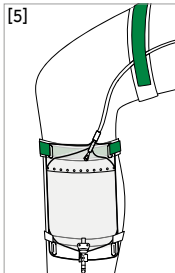
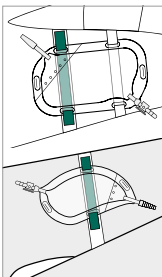
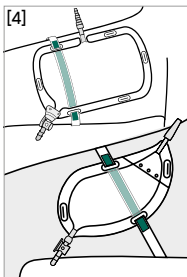
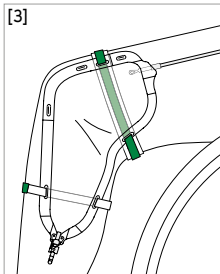
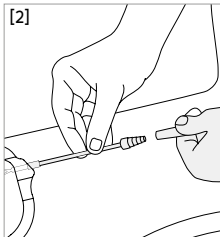
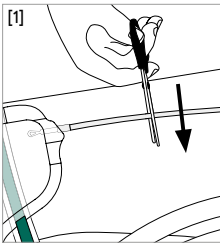
English

Sterile SAUER urine bag

Instructions on how to use sterile urine bags should be provided by a medical expert. Details of the device's trade name and performance characteristics can be found on the label. Functional bags can be used as leg and/or night bags; bendi, leg, sport, discreet and rectangular bags are designed for use as leg bags. We recommend our leg straps (REF 57.44, 57.48, 57.49) (accessories) for attaching the bags to the leg and our retaining straps (REF 57.38 and 57.39) for securing the inlet tubes to reduce the risk of the tube twisting or pulling on the drainage system.

Intended use and indications

Sterile urine bags serve to collect the urine of persons with an invasive urine drainage system (e.g. transurethral indwelling catheter, suprapubic catheter, etc.).

Risks and warnings

The device must a) not be reused and b) not be used if the sterile barrier is damaged in order not to increase the risk of infection.

Urine bags must not be attached to the leg in the case of peripheral circulatory disorders, diabetic foot syndrome, open leg etc., as there is a risk of infection. Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening urosepsis/autonomic dysreflexia. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention. If serious incidents occur when using "SAUER" urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.

Precautions during use

- To reduce sources of infection, do not touch the adaptor of the sterile leg/night bag or bring it into contact with surfaces after removing the protective cap. Connect straight away. To cut the inlet tube, use only scissors that have been disinfected beforehand (use a suitable disinfectant, e.g. for surfaces)[1].
- To ensure problem-free urine drainage, attach the leg or night bag below the bladder and avoid kinking or twisting of the inlet tube. With leg bags, straps should be secured under the bag.
- To ensure adequate circulation, do not over-tighten the leg bag straps and check the area under the straps regularly for skin changes, line with a soft pad if necessary.
- Always disconnect the bag before opening the belt.
- Always connect without tension.
- Replace belt if there are signs of wear and tear.

Leg bag instructions for use (Fig. [1]–[6] see p. 2)

The leg bag can be worn on either the left- or right-hand side. The right leg is recommended for leg bags with a white swing tap and the left leg for leg bags with a yellow swing tap in order to avoid accidental opening due to the tap wing resting on the side of the leg. To secure the bag to the leg, thread a leg strap provided for this purpose through two eyelets on the bag. Fasten the strap lying flat between the bag and leg. Depending on the bag volume, use a second strap [4, 5].

[1] When using the included stepped adaptor, cut the inlet tube to the desired length (straight cut). Attach the adaptor without touching it.

[2] To connect the leg bag to a catheter, remove the protective cap of the inlet tube adaptor and push it into the funnel of the urine drain without touching it.

[3] Bendi bag/Bendi bag with needle-free sample port: Multiple eyelets allow the bag to be attached at the knee

[4] Sport/Discreet bag: Attachment to the thigh (inside or outside) or inside the trouser leg without fixation.

[3] [4] [5] Check that it is securely in place.

[6] Leg/rectangular/functional bag: Attachment to the thigh or lower leg. If necessary, secure the inlet tube using a retaining strap.

Night bag instructions for use (Fig. [7], [8] see p. 2)

[7] After removing from the packaging, roll out the inlet tube as shown to smooth it down.

[8] Then hang the bag on the bed frame or on a separate stand using a bed holder. Finally, remove the protective cap from the adaptor and push this into the funnel of the indwelling urine drainage system without touching it.

Removal and changing

To empty the bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again. To change a bag, carefully disconnect from the indwelling urine drainage system without touching the open ends of the connections.

Disposal

Empty before disposal. Dispose of the urine bag in the residual waste.









	Catalog number
	Store in a dry place
	Do not resterilise
	Quantity
	Keep away from sunlight
	Swiss authorised representative
	UK authorised representative
	Sterilised with ethylene oxide
	Single sterile barrier system
	Batch number
	Observe instructions for use
	CE marked medical device
	Use by (year-month-day)
	Not for reuse
	Pharmaceutical central no.
	Drug Tariff
	List of products and services
	Do not use if package is damaged
	Date of manufacture
	Manufacturer

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

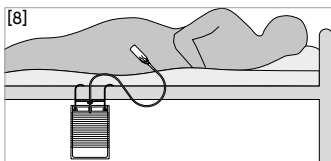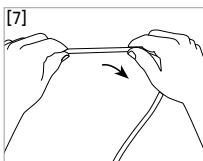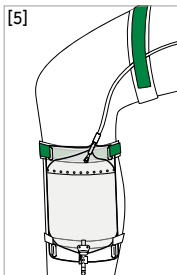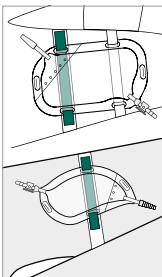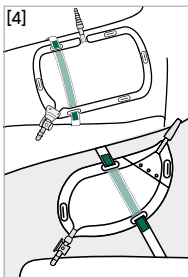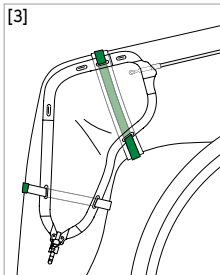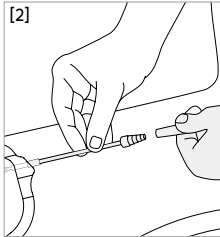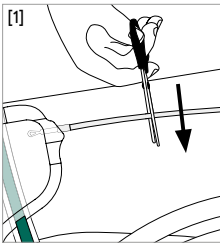Should you have any questions or problems, please turn to a medical professional or to our customer adviser:

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