
SAUER-Urinbeutel

Steril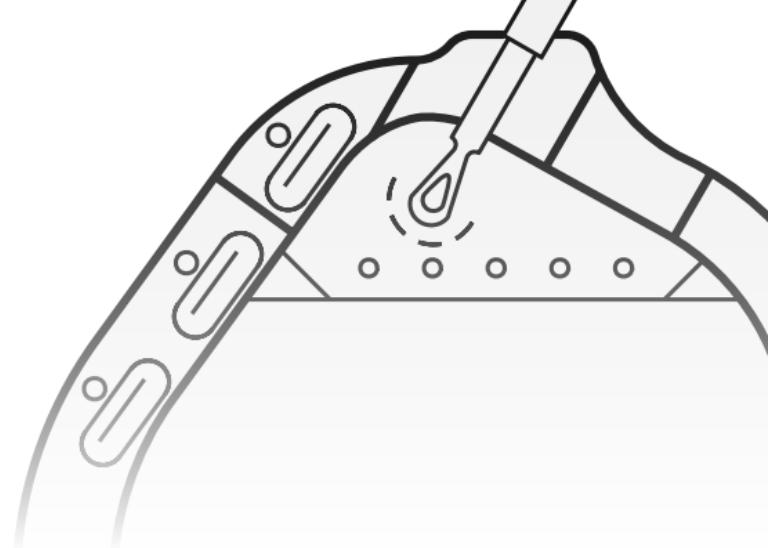

Sterile SAUER-Urinbeutel	Deutsch	2
Sterile SAUER urine bag	English	4
Poches urinaires stériles SAUER	Français	6
Sacca sterile per drenaggio urinario SAUER	Italiano	8
Steriele urineopvangzakken SAUER	Nederlands	10
Steril SAUER-urinpåse	Svenska	12
Sterile SAUER-urinposer	Dansk	14
Steriili SAUER-virtsapussi	Suomi	16
Sterile Sauer-urinposer	Norsk	18
Steriilsed SAUER-uriinikotid	Eesti	20
Bolsa colectora de orina estéril SAUER	Español	22
Sacos de Estéreis SAUER	Português	24
Αποστειρωμένος ουροσυλλέκτης SAUER	Ελληνική	26
Steril SAUER vizeletgyűjtő zsák	Magyar	28
Sterylny worek na mocz SAUER	Polski	30
Sterilní urinální sáčky SAUER	Čeština	32
Pungă de colectare a urinei SAUER sterilă	Română	34
Стерилни уринаторни торбички SAUER	Български	36
Sterilne urinske vrećice SAUER	Slovenščina	38
Sterilne urinske vrećice SAUER	Hrvatski	40
Sterilne SAUER urin kese	Srpski	42
Sterilne SAUER vrećice za urin	Bosanski	44
Стерилна SAUER-уринарна кеса	Македонски	46
49	فارسی	کیسه های ادرار SAUER
51	العربية	أكياس البول شركة زاور

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Instructions for use – sterile SAUER urine bag

Instructions on how to use sterile urine bags should be provided by a medical expert. Details of the device's trade name and performance characteristics can be found on the label. Functional bags can be used as leg and/or night bags; bendi, leg, sport, discreet and rectangular bags are designed for use as leg bags.

Intended use and indications

Sterile urine bags serve to collect the urine of persons with an invasive urine drainage system (e.g. transurethral indwelling catheter, suprapubic catheter, etc.).

Risks and warnings

The device must a) not be reused and b) not be used if the sterile barrier is damaged in order not to increase the risk of infection.

Urine bags must not be attached to the leg in the case of peripheral circulatory disorders, diabetic foot syndrome, open leg etc., as there is a risk of infection.


Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening urosepsis/autonomic dysreflexia. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention.

Precautions during use


- To reduce sources of infection, do not touch the adaptor of the sterile leg/night bag or bring it into contact with surfaces after removing the protective cap. Connect straight away. To cut the inlet tube, use only scissors that have been disinfected beforehand (use a suitable disinfectant, e.g. for surfaces)[1].
- To ensure problem-free urine drainage, attach the leg or night bag below the bladder and avoid kinking or twisting of the inlet tube. With leg bags, straps should be secured under the bag.
- To ensure adequate circulation, do not over-tighten the leg bag straps and check the area under the straps regularly for skin changes, line with a soft pad if necessary.
- Always disconnect the bag before opening the belt.
- Always connect without tension.
- Replace belt if there are signs of wear and tear.

Leg bag instructions for use


The leg bag can be worn on either the left- or right-hand side. The right leg is recommended for leg bags with a white swing tap and the left leg for leg bags with a yellow swing tap in order to avoid accidental opening due to the tap wing resting on the side of the leg. To connect the leg bag to a catheter, remove the protective cap of the inlet tube adaptor and push it into the funnel of the urine drain without touching it [2]. Check that it is securely in place [3] [4] [5].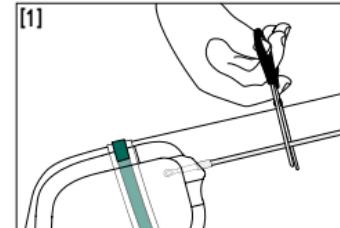

When using the included stepped adaptor, cut the inlet tube to the desired length (straight cut). Attach the adaptor.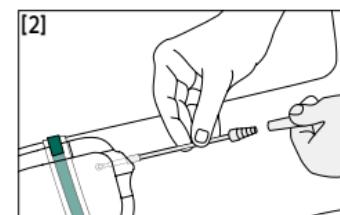



Remove the protective cap from the adapter and connect without touching.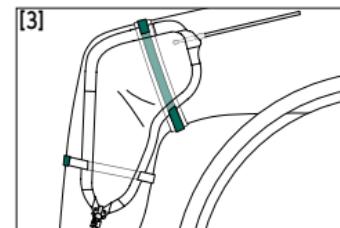

Bendi bag/Bendi bag with needle-free sample port:
Multiple eyelets allow the bag to be attached at the knee.



Sport/Discreet bag: Attachment to the thigh (inside or outside) or inside the trouser leg without fixation.

Leg/rectangular/functional bag: Attachment to the thigh or lower leg.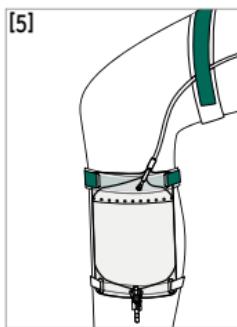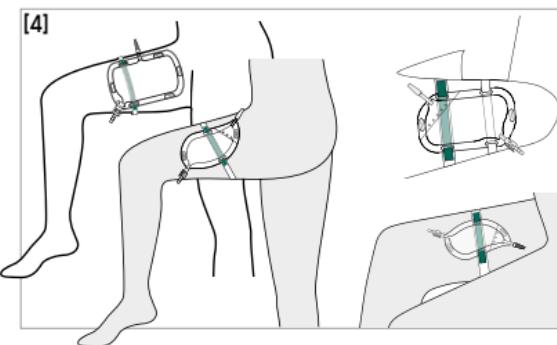

Night bag instructions for use

After removing from the packaging, roll out the inlet tube as shown to smooth it down [6]. Then hang the bag on the bed frame or on a separate stand using a bed holder [7]. Finally, remove the protective cap from the adaptor and push this into the funnel of the indwelling urine drainage system without touching it.

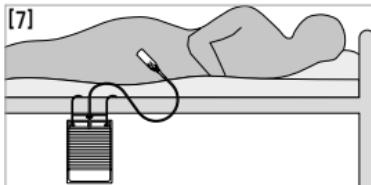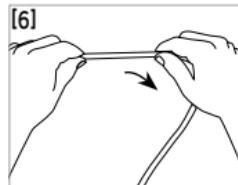Removal and changing

To empty the bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again. If there is no outlet valve, cut off the top corner and empty the urine into a toilet.

To change a bag, carefully disconnect from the indwelling urine drainage system without touching the open ends of the connections.

Disposal

Empty before disposal. Dispose of the urine bag in the residual waste.

	Article number		Pharmaceutical central no.
	Store in a dry place		Drug Tariff
	Do not resterilise		List of products and services
	Quantity		Do not use if package is damaged
	Keep away from sunlight		Date of manufacture
	Swiss authorised representative		Manufacturer
	Sterilised with ethylene oxide		Importer
	Single sterile barrier system	If serious incidents occur when using "SAUER" urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.	
	Batch number	MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.	
	Observe instructions for use		
	CE marked 0123 medical device		
	Use by (year-month-day)		
	Not for reuse		

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