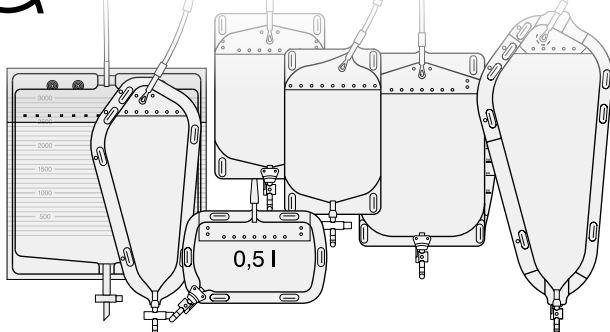
MANFRED SAUER

SAUER-Urinbeutel

NON STERILE

Abbildungen/Figures/Illustraties/Kuva/Illusztrációk/موسرلل/موسرلل/موسرلل

2

Gebrauchsanweisung	Deutsch	3
Instructions for use	English	5
Notice d'utilisation	Français	7
Istruzioni per l'uso	Italiano	9
Gebruiksaanwijzing	Nederlands	11
Bruksanvisning	Svenska	13
Brugsanvisning	Dansk	15
Käyttöohje	Suomi	17
Bruksanvisning	Norsk	19
Notkunarleiðbeiningar	Ísland	21
Kasutusjuhised	Eesti	23
Instrucciones de uso	Español	25
Instruções de utilização	Português	27
Οδηγίες χρήσης	Ελληνική	29
Használati utasítás	Magyar	31
Instrukcja użytkowania	Polski	33
Návod k použití	Čeština	35
Instrucțiuni de utilizare	Română	37
Инструкция за употреба	Български	39
Navodila za uporabo	Slovenščina	41
Upute za uporabu	Hrvatski	43
Uputstvo za upotrebu	Српски/srpski	45
Uputstvo za upotrebu	Bosanski	47
Упатство за употреба	Македонски	49
دستور العمل استفاده	فارسی	51
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	53








MANFRED SAUER GmbH

Neurott 7-15 · 74931 Lobbach · Germany
Tel. +49 6226 960 100 · info@manfred-sauer.de
www.manfred-sauer.com

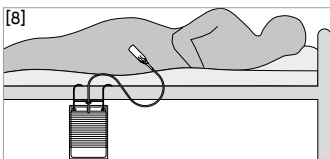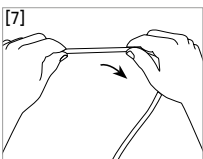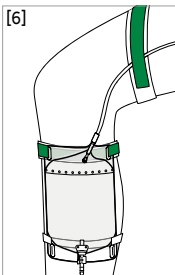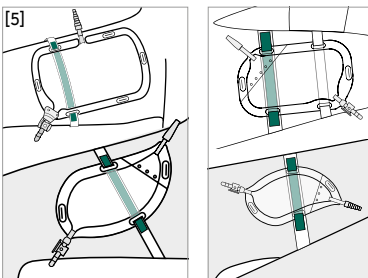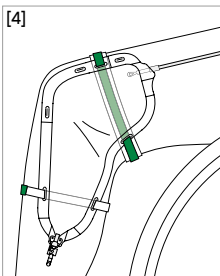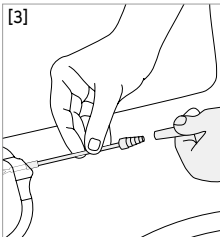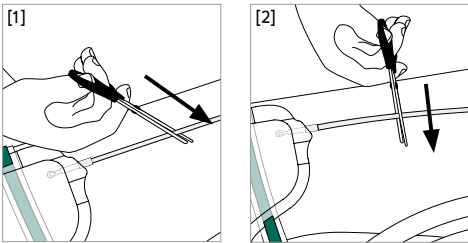20260217 \$22772

Deutsch

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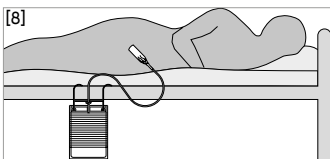
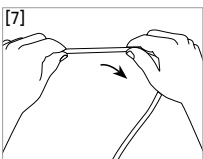
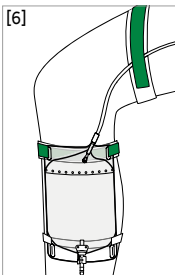
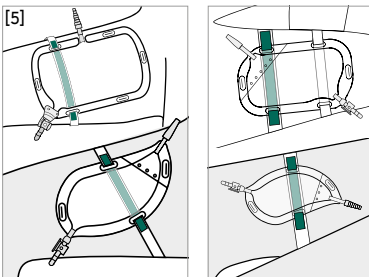
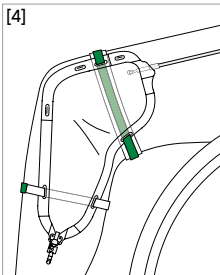
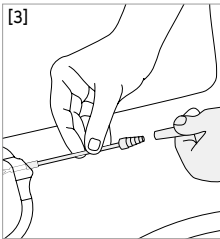
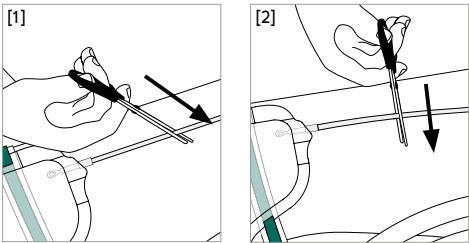
English

Non-sterile SAUER urine bag

Details of the device's trade name and performance characteristics can be found on the label. Functional bags can be used as leg and/or night bags; bendi, leg, sport, discreet and rectangular bags are designed for use as leg bags. For single use only. We recommend our leg straps (REF 57.44, 57.48, 57.49) (accessories) for attaching the bags to the leg and our retaining straps (REF 57.38 and 57.39) for securing the inlet tubes to reduce the risk of the tube twisting or pulling on the drainage system.

Intended use and indications

Urine collection bags serve to collect the urine of persons with a urine drainage system.

Contraindications

Given the increased risk of infection and/or other complications, non-sterile bags must not be connected to an invasive indwelling urine drainage system (e.g. transurethral indwelling catheter, suprapubic catheter, etc.).

Risks and warnings

Urine bags must not be attached to the leg in the case of peripheral circulatory disorders, diabetic foot syndrome, open leg etc.

Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening uroperitonitis/autonomic dysreflexia. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention.

If serious incidents occur when using SAUER urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.

Precautions during use

- To ensure problem-free urine drainage, attach the leg or night bag below the bladder and avoid kinking or twisting of the inlet tube. With leg bags, straps should be secured under the bag.
- To ensure adequate circulation, do not over-tighten the leg bag straps and check the area under the straps regularly for skin changes, line with a soft pad if necessary.
- Replace belt if there are signs of wear and tear.

Leg bag instructions for use (Fig. [1]–[6] see p. 2):

The leg bag can be worn on either the left- or right-hand side. The right leg is recommended for leg bags with a white swing tap and the left leg for leg bags with a yellow swing tap in order to avoid accidental opening due to the tap wing resting on the side of the leg. To secure the bag to the leg, thread a leg strap provided for this purpose through two eyelets on the bag. Fasten the strap lying flat between the bag and leg. Depending on the bag volume, use a second strap.

[1] Cut the inlet tube of the bendi bag at an angle to suit the wearer's height in order to make it easier to push on the sheath tube. Moisten if necessary.

[2] Straight cut if the inlet tube is connected to an adaptor.

[3] To connect the leg bag to a urinary sheath, push the end of the tube or adapter of the bag into the connecting tube of the sheath.

[4] Bendi bag: Multiple eyelets allow the bag to be attached at the knee.

[5] Sport/paediatric, discreet bag: Attachment to the thigh (inside or outside) or inside the trouser leg without fixation.

[4][5][6] Ensure it is securely in place. If necessary, secure the inlet tube using a retaining strap.

[6] Leg/rectangular/functional bag: Attachment to the thigh or lower leg.

Night bag instructions for use (Fig. [7], [8] see p. 2):

[7] After removing from the packaging, roll out the inlet tube as shown to smooth it down.

[8] Then hang the bag on the bed frame or on a separate stand using a bed holder. Finally, remove the protective cap from the adaptor and push this into the funnel of the indwelling urine drainage system without touching it.

Connecting a leg or night bag

A night bag can be connected to a leg bag for increased volume at night. For this, a connecting tube is pushed onto the outlet tap of the leg bag and connected to the night bag using an adaptor. In this case, the leg bag seal remains open.

Removal and changing

To empty the bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again. To change a bag, carefully disconnect from the indwelling urine drainage system without touching the open ends of the connections.








Disposal

Empty before disposal. Dispose of the urine bag in the residual waste.

REF	Catalog number
☂	Store in a dry place
📦	Quantity
☀	Keep away from sunlight
CH REP	Swiss authorised representative
UK REP	UK authorised representative
LOT	Batch number
📖 i	Observe instructions for use
CE MD	CE marked medical device
🕒	Use by (year-month-day)
🔄	Not for reuse
PZN	Pharmaceutical central no.
DT	Drug Tariff
LPP	List of products and services
🏭	Manufacturer

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

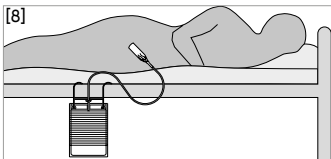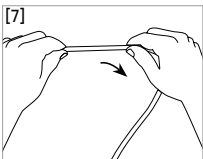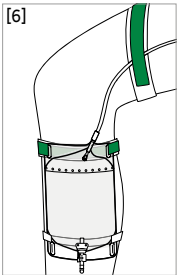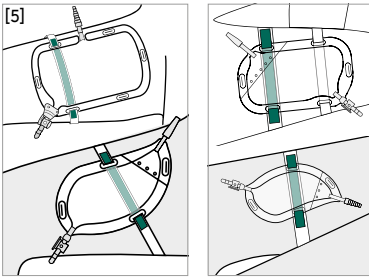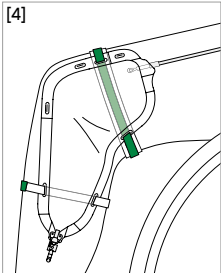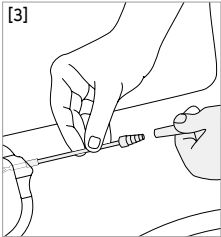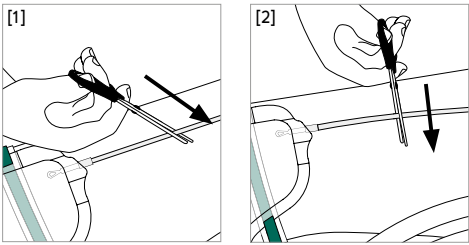Should you have any questions or problems, please turn to a medical professional or to our customer adviser: **MANFRED SAUER GmbH · Neurott 7–15 · 74931 Lobbach · Germany · +49 6226 960100**

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