
SAUER-Urinbeutel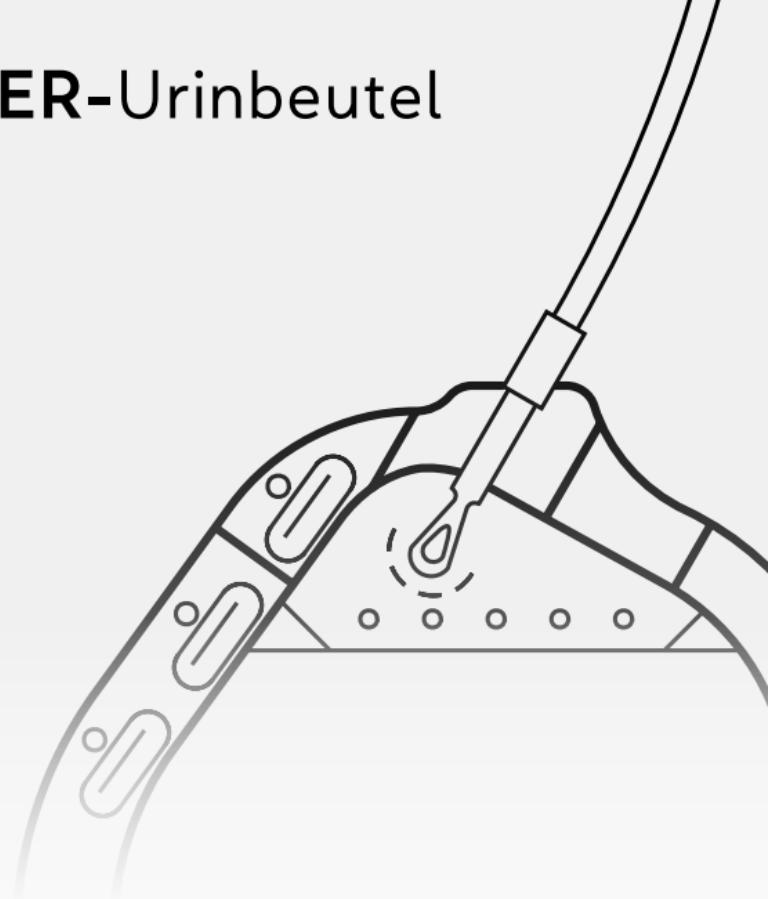

SAUER-Urinbeutel	Deutsch	2
SAUER urine bag	English	4
Poches urinaires SAUER	Français	6
Sacca per drenaggio urinario SAUER	Italiano	8
Urineopvangzakken SAUER	Nederlands	10
SAUER-urinpåse	Svenska	12
SAUER-urinposer	Dansk	14
SAUER-virtsapussi	Suomi	16
Sauer-urinposer	Norsk	18
SAUER-uriinikotid	Eesti	20
Bolsa colectora de orina SAUER	Español	22
Sacos de SAUER	Português	24
Ουροσυλλέκτης SAUER	Ελληνική	26
SAUER vizeletgyűjtő zsák	Magyar	28
Worek na mocz SAUER	Polski	30
Urinální sáčky SAUER	Čeština	32
Pungă de colectare a urinei SAUER	Română	34
Уринаторни торбички SAUER	Български	36
Urinske vrečke SAUER	Slovenščina	38
Urinske vrećice SAUER	Hrvatski	40
SAUER urin kese	Srpski	42
SAUER vrećice za urin	Bosanski	44
SAUER-уринарна кеса	Македонски	46
49	فارسی	SAUER کیسه های ادرار
51	العربية	أكياس البول شركة زاور

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Tel. +49 6226 960 100 · info@manfred-sauer.de

www.manfred-sauer.com

Instructions for use – non-sterile SAUER urine bag

Details of the device's trade name and performance characteristics can be found on the label. Functional bags can be used as leg and/or night bags; bendi, leg, sport, discreet and rectangular bags are designed for use as leg bags. For single use only.

Intended use and indications

Urine collection bags serve to collect the urine of persons with a urine drainage system.

Contraindications

Given the increased risk of infection and/or other complications, non-sterile bags must not be connected to an invasive indwelling urine drainage system (e.g. transurethral indwelling catheter, suprapubic catheter, etc.).

Risks and warnings

Urine bags must not be attached to the leg in the case of peripheral circulatory disorders, diabetic foot syndrome, open leg etc.



Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening urosepsis/autonomic dysreflexia. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention.

Precautions during use



- To ensure problem-free urine drainage, attach the leg or night bag below the bladder and avoid kinking or twisting of the inlet tube. With leg bags, straps should be secured under the bag.
- To ensure adequate circulation, do not over-tighten the leg bag straps and check the area under the straps regularly for skin changes, line with a soft pad if necessary.
- Replace belt if there are signs of wear and tear.

Leg bag instructions for use

The leg bag can be worn on either the left- or right-hand side. The right leg is recommended for leg bags with a white swing tap and the left leg for leg bags with a yellow swing tap in order to avoid accidental opening due to the tap wing resting on the side of the leg. Cut the inlet tube of the bendi bag at an angle to suit the wearer's height in order to make it easier to push on the sheath tube [1]. Moisten if necessary. Straight cut if the inlet tube is connected to an adaptor [2]. To connect the leg bag to a urinary sheath, push the end of the tube or adapter of the bag into the connecting tube of the sheath [3]. Ensure it is securely in place [4][5][6].

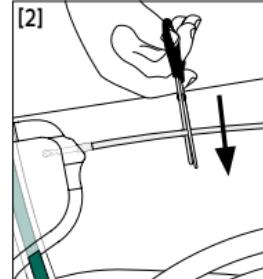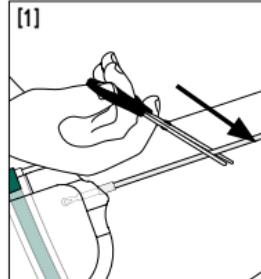
Use scissors to cut the inlet tube to the desired length (left: angled cut with a hard tube, right: straight cut with a soft tube).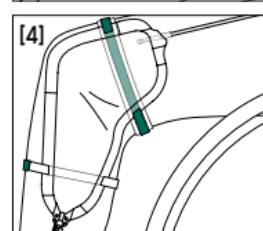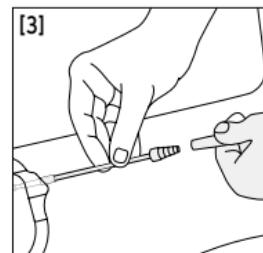


Bendi bag: Multiple eyelets allow the bag to be attached at the knee.

Sport/paediatric, discreet bag: Attachment to the thigh (inside or outside) or inside the trouser leg without fixation.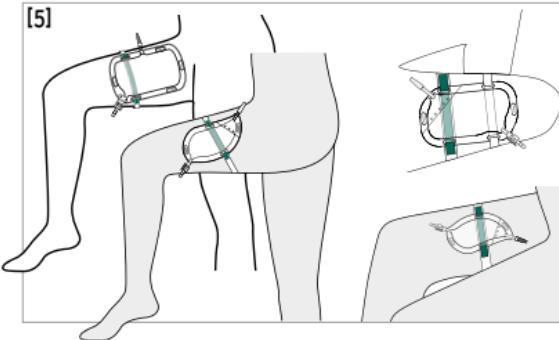



Leg/rectangular/functional bag: Attachment to the thigh or lower leg.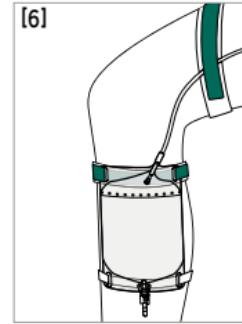

Night bag instructions for use

After removing from the packaging, roll out the inlet tube as shown to smooth it down [7]. Then hang the bag on the bed frame or on a separate stand using a bed holder [8]. Finally, remove the protective cap from the adaptor and push this into the funnel of the indwelling urine drainage system without touching it.

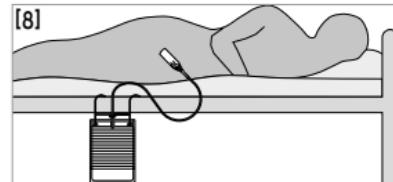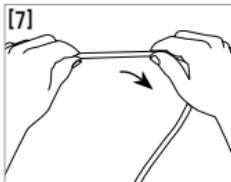Connecting a leg or night bag

A night bag can be connected to a leg bag for increased volume at night. For this, a connecting tube is pushed onto the outlet tap of the leg bag and connected to the night bag using an adaptor. In this case, the leg bag seal remains open.

Removal and changing

To empty the bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again. If there is no outlet valve, cut off the top corner and empty the urine into a toilet.

To change a bag, carefully disconnect from the indwelling urine drainage system without touching the open ends of the connections.

Disposal

Empty before disposal. Dispose of the urine bag in the residual waste.

	Article number		Do not use if package is damaged
	Store in a dry place		Date of manufacture
	Quantity		Manufacturer
	Keep away from sunlight		Importer
	Swiss authorised representative		
	Batch number		
	Observe instructions for use		
	CE marked medical device		
	Use by (year-month-day)		
	Not for reuse		
	Pharmaceutical central no.		
	Drug Tariff		
	List of products and services		
If serious incidents occur when using "SAUER" urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.			
MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.			
	MANFRED SAUER GmbH Neurott 7-15 · 74931 Lobbach · Germany Tel. +49 6226 960 104 · export@manfred-sauer.de www.manfred-sauer.com		

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