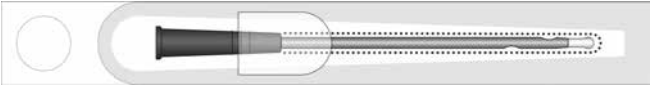
IQ2302.xx

iQ CATH

Gebrauchsanweisung	Deutsch	2
Instructions for use	English	4
Notice d'utilisation	Français	6
Istruzioni per l'uso	Italiano	8
Gebruiksaanwijzing	Nederlands	10
Bruksanvisning	Svenska	12
Brugsanvisning	Dansk	14
Käyttöohje	Suomi	16
Bruksanvisning	Norsk	18
Kasutusjuhised	Eesti	20
Instrucciones de uso	Español	22
Instruções de utilização	Português	24
Οδηγίες χρήσης	Ελληνική	26
Használati utasítás	Magyar	28
Instrukcja użytkowania	Polski	30
Návod k použití	Čeština	32
Instrucțiuni de utilizare	Română	34
Инструкция за употреба	Български	36
Navodila za uporabo	Slovenščina	38
Upute za uporabu	Hrvatski	40
Uputstvo za upotrebu	Српски/srpski	42
Uputstvo za upotrebu	Bosanski	44
Упатство за употреба	Македонски	46
دستور العمل استفاده	فارسی	48
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	50

Deutsch

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www.manfred-sauer.com

English

Hydrophilic Single-use catheter for ISC (Intermittent self-catheterisation)

- Immediately ready for use
- The catheter with 2 functional zones (rounded tip and guiding zone)
- Peel packaging for easy handling
- Total length: ca. 170 mm, length without funnel: ca. 120 mm
- 3 sizes: CH 10, CH 12, CH 14

Group of Users: IQ2302.xx – Women, girls

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra, obstructions (e.g. tumours) in the urethra, at the sphincter blocking passage.

Instructions for use:

The first use should be supervised by a professional.


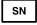
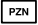
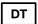
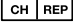
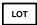
To have your hands free during disinfection, the catheter can be attached to a suitable surface with the adhesive dot.

Risks and Cautions:


The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.

Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without activating with sterile saline solution 0.9% first as it is not lubricated. Tap water affects catheter sterility and thus poses an increased infection risk. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C.

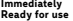
Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

	Catalog number		CE marked medical device
	Serial number		Use by (year-month-day)
	Store in a dry place		Not for reuse
	This way up		Pharmaceutical central no.
	Do not resterilise		Drug Tariff
	Quantity		List of products and services
	Keep away from sunlight		Do not use if package is damaged
	Sterilised with ethylene oxide		Date of manufacture
	Single sterile barrier system		Manufacturer
	Single sterile barrier system with protective packaging outside		Swiss authorised representative
	Batch number		Rounded tip
	Observe instructions for use		Immediately ready for use

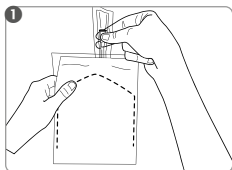
Important! Aluminium pouches containing 10 catheters each provide an additional layer of protection against drying up. Condensation may form in these pouches occasionally, which does not affect the quality of the product. Use the catheters within 30 days of opening the aluminium pouch.

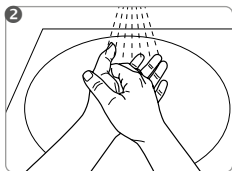
Open the aluminium bag and take one catheter.

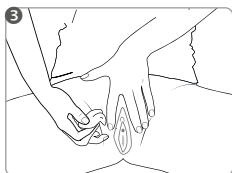
Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly.

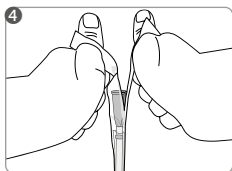
Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

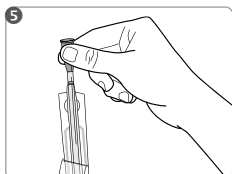
Hold the catheter vertically with the tip downwards and open the "Peel" packaging at the funnel end.

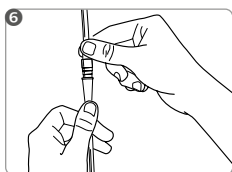Remove the catheter from the plastic peel film holding it at the funnel. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

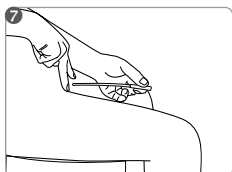If necessary, attach a urine bag to the catheter.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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