MANFRED SAUER

***iQ*CATH** IQ3004.xx

Abbildungen/Figures/Illustraties/Kuva/Illusztrációk/ةحيضوتلا مودرلا 2

Gebrauchsanweisung	Deutsch	3
Instructions for use	English	5
Notice d'utilisation	Français	7
Istruzioni per l'uso	Italiano	9
Gebruiksaanwijzing	Nederlands	11
Bruksanvisning	Svenska	13
Brugsanvisning	Dansk	15
Käyttöohje	Suomi	17
Bruksanvisning	Norsk	19
Notkunarleiðbeiningar	Ísland	21
Kasutusjuhised	Eesti	23
Instrucciones de uso	Español	25
Instruções de utilização	Português	27
Οδηγίες χρήσης	Ελληνική	29
Használati utasítás	Magyar	31
Instrukcja użytkowania	Polski	33
Návod k použití	Čeština	35
Instrucţiuni de utilizare	Română	37
Инструкция за употреба	Български	39
Navodila za uporabo	Slovenščina	41
Upute za uporabu	Hrvatski	43
Uputstvo za upotrebu	Српски/srpski	45
Uputstvo za upotrebu	Bosanski	47
Упатство за употреба	Македонски	49
دستور العمل استفادہ	فارسی	51
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	53










MANFRED SAUER GmbH

Neurott 7-15 · 74931 Lobbach · Germany
Tel. +49 6226 960 100 · info@manfred-sauer.de
www.manfred-sauer.com

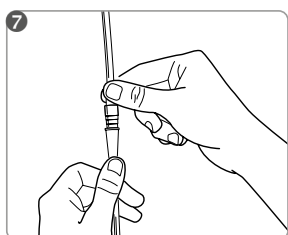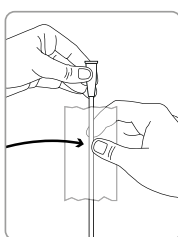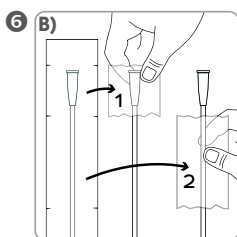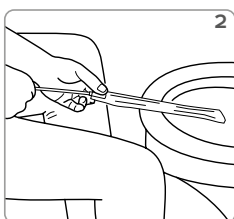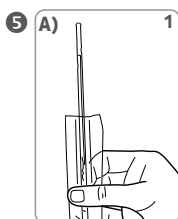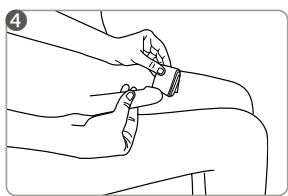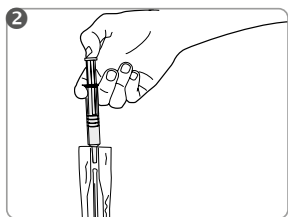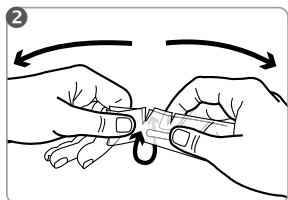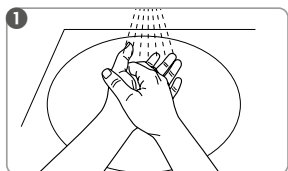20260211 516801

Deutsch

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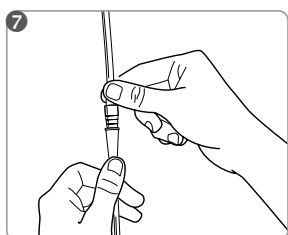
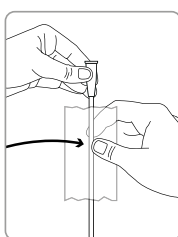
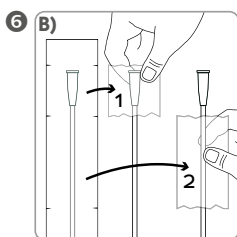
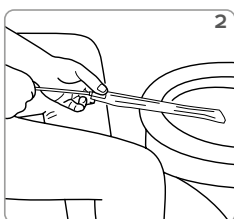
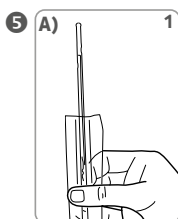
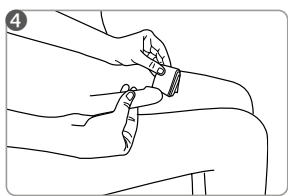
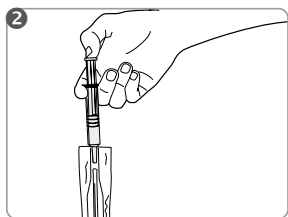
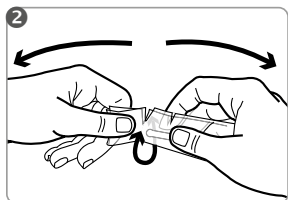
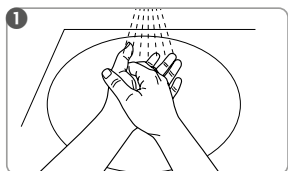
MANFRED SAUER GmbH · Neurott 7–15 · 74931 Lobbach · Germany · +49 6226 960100

English

Single use gel catheter for ISC
(Intermittent self-catheterisation)

- Not ready for use
- The catheter with 2 functional zones (SAUER-rounded tip and guiding zone)
- Functional packaging for variable use
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 3 sizes: CH 12, CH 14, CH 16

Group of Users: IQ3004.xx – Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:

The first use should be supervised by a professional. With gel catheters, there may sometimes be a slight delay after insertion before the urine begins to flow. Ensure sufficient lighting during insertion, adjustment and removal.

Risks and Cautions:

The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs. Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel as it is not lubricated. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C. Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

	Catalog number
	Serial number
	Store in a dry place
	This way up
	Do not resterilise
	Quantity
	Keep away from sunlight
	Sterilised with ethylene oxide
	Single sterile barrier system
	Single sterile barrier system with protective packaging outside
	Batch number
	Observe instructions for use
	CE marked medical device
	Use by (year-month-day)
	Not for reuse
	Pharmaceutical central no.
	Drug Tariff
	List of products and services
	Do not use if package is damaged
	Date of manufacture
	Manufacturer
	Swiss authorised representative
	UK authorised representative
	SAUER-rounded tip
	Not ready for use

Fig. 1-7 (see p. 2): Avoid touching any part of the catheter surface (except for the connector) directly. You can use the functional packaging as a gripping aid.

- 1 Before use, wash hands thoroughly and disinfect them.
- 2 Use tear marks to open near the catheter tip and tear off with a jerk.
- 3 Fill in the sterile lubricating gel and spread along the full length of the catheter (except for the funnel) to make sure all of the surface is coated with gel.
- 4 Hold the catheter vertically with the tip down and fill in a sufficient quantity of sterile saline solution 0.9% to ensure that all of the catheter surface (except for the funnel) is moistened for at least 30 sec.

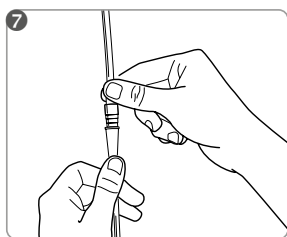
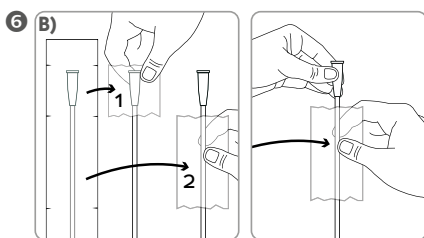
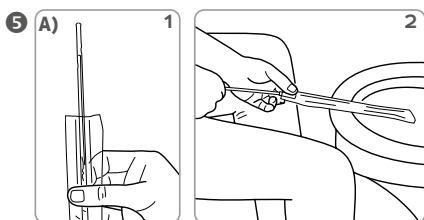
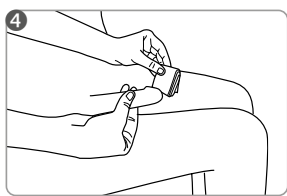
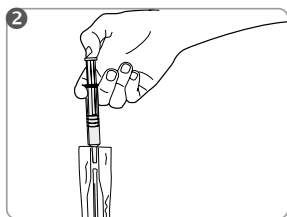
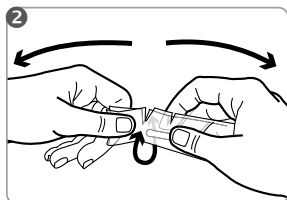
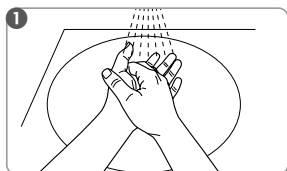
Using the functional packaging

- 5 A) The functional packaging can be used as an extension for draining the urine directly into the toilet. To do so, tear off the functional packaging along the front tear marks (catheter tip). Then slide the catheter forwards (Fig. 5.1) until the connector seals the packaging (Fig. 5.2).
- 6 (B) The functional packaging can also be used as a short or long gripping aid (1 or 2). To do so, remove the required section of film along the corresponding tear marks.
- 7 Alternatively, you can hold the catheter at the connector and remove the functional packaging completely. Attach a urine bag to the catheter if necessary. If the catheter becomes stuck inside the packaging, twist the catheter slowly to remove. Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter in the general household waste.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser:
MANFRED SAUER GmbH · Neutrott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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