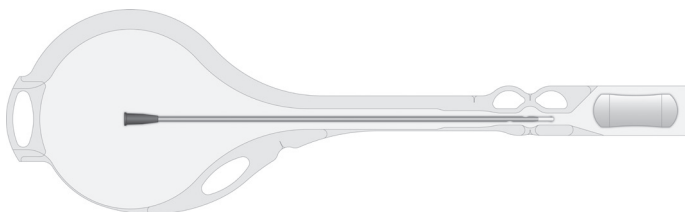
MANFRED SAUER

***IQCATH*****IQ3604.xx | IQ3614.xx**

Abbildungen/Figures/Illustraties/Kuva/Illusztrációk/ةيحوضوتلا مونسرلا


2

Gebrauchsanweisung	Deutsch	3
Instructions for use	English	5
Notice d'utilisation	Français	7
Istruzioni per l'uso	Italiano	9
Gebruiksaanwijzing	Nederlands	11
Bruksanvisning	Svenska	13
Brugsanvisning	Dansk	15
Käyttöohje	Suomi	17
Bruksanvisning	Norsk	19
Notkunarleiðbeiningar	Ísland	21
Kasutusjuhised	Eesti	23
Instrucciones de uso	Español	25
Instruções de utilização	Português	27
Οδηγίες χρήσης	Ελληνική	29
Használati utasítás	Magyar	31
Instrukcja użytkowania	Polski	33
Návod k použití	Čeština	35
Instrucţiuni de utilizare	Română	37
Инструкция за употреба	Български	39
Navodila za uporabo	Slovenščina	41
Upute za uporabu	Hrvatski	43
Uputstvo za upotrebu	Српски/srpski	45
Uputstvo za upotrebu	Bosanski	47
Упатство за употреба	Македонски	49
دستور العمل استفادہ	فارسی	51
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	53

**MANFRED SAUER GmbH**

Neurott 7-15 · 74931 Lobbach · Germany
Tel. +49 6226 960 100 · info@manfred-sauer.de
www.manfred-sauer.com

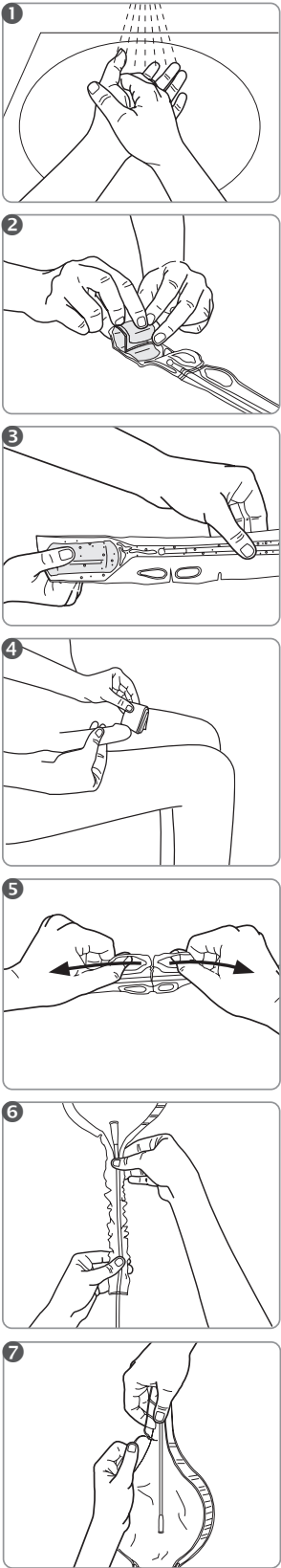20260205 516832

Deutsch

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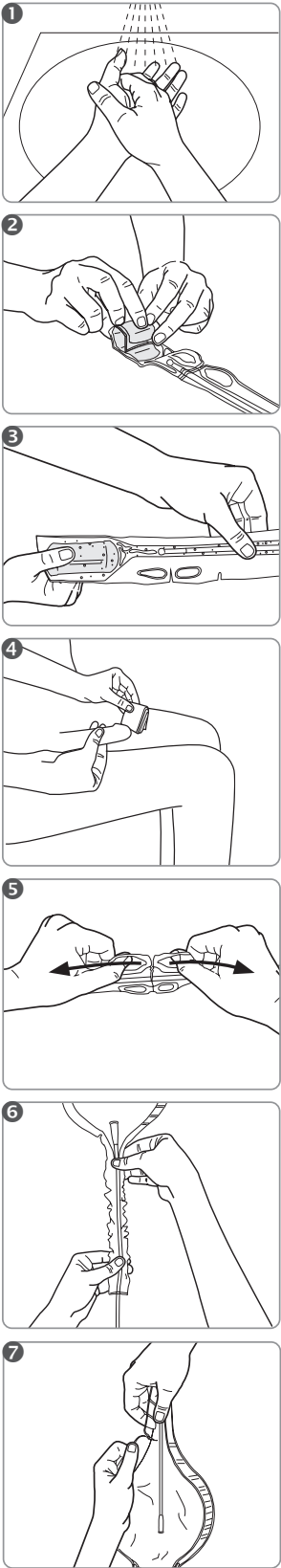
English

Single use gel catheter for ISC
(Intermittent self-catheterisation)

- Ready for use
- Catheter with 2 functional zones (SAUER-rounded tip and guiding zone)
- Urine collection bag
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 3 sizes: CH 12, CH 14, CH 16

Group of Users: IQ36x4.xx – Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:

The first use should be supervised by a professional.

If liquid has already leaked from the gel sachet into the packaging, the catheter may no longer be used.

With gel catheters, there may sometimes be a slight delay after insertion before the urine begins to flow. Ensure sufficient lighting during insertion, adjustment and removal.

Risks and Cautions:

The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs. Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel as it is not lubricated. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C.

Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

Fig. 1-7 (see p. 2): Avoid touching any part of the catheter surface (except for the connector) directly.


- 1 Before use, wash hands thoroughly and disinfect them.
- 2 Open the gel sachet by folding and squeezing. Make sure the whole content of the sachet is emptied into the packaging.
- 3 Spread the gel along the full length of the catheter (except for the funnel).
- 4 Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).
- 5 Open the urine bag at the front end using the tabs and release the tip of the catheter. Open by pulling sharply sideways. If the catheter becomes stuck inside the packaging, release the catheter by turning it at the connector and twist slowly to remove.
- 6 Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.
- 7 Push back the catheter into the urine bag and seal this by making a knot. To empty the bag tear at the back end.

Dispose of the catheter in the general household waste.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

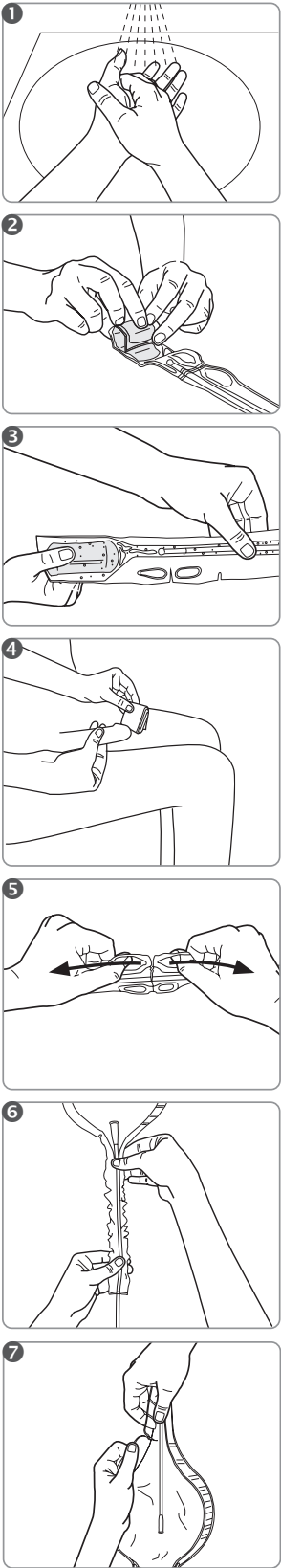REF	Catalog number
SN	Serial number
	Store in a dry place
	This way up
	Do not resterilise
	Quantity
	Keep away from sunlight
STERILE EO	Sterilised with ethylene oxide
	Single sterile barrier system
	Single sterile barrier system with protective packaging outside
LOT	Batch number
	Observe instructions for use
CE MD 0123	CE marked medical device
	Use by (year-month-day)
	Not for reuse
PZN	Pharmaceutical central no.
DT	Drug Tariff
LPP	List of products and services
	Do not use if package is damaged
	Date of manufacture
	Manufacturer
CH REP	Swiss authorised representative
UK REP	UK authorised representative
	SAUER-rounded tip
Ready for use	Ready for use

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