KIC-SYSTEM

KIC System	Deutsch	2
KIC System	English	4
Système KIC	Français	6
Sistema KIC	Italiano	8
KIC-systeem	Nederlands	10
KIC-systemet	Svenska	12
KIC-system	Dansk	14
KIC-järjestelmä	Suomi	16
KIC-system	Norsk	18
KIC-búnaður	Ísland	20
KIC-süsteem	Eesti	22
Sistema KIC	Español	24
Sistema KIC	Português	26
Σύστημα KIC	Ελληνική	28
KIC rendszer	Magyar	30
System KIC	Polski	32
Systém KIC	Čeština	34
Sistemul KIC	Română	36
Система KIC	Български	38
Sistem KIC	Slovenščina	40
Sustav KIC	Hrvatski	42
KIC sistem	Srpski	44
KIC sistem	Bosanski	46
51	فارسی	سیستم KIC
53	العربية	نظام كيه أي سي

KIC System (KIC-Kondome, KIC-Spreizring, KIC-Kupplung)

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KIC System (KIC sheaths, KIC sheath expander, KIC connector)

Intended use and indication

KIC urinary sheaths are used for external urinary drainage into a urine collection bag in men with urinary incontinence with residual urine that is treated additionally with regular intermittent self or assisted catheterisation (ISC/IAC). They are used on intact skin. The KIC system, consisting of KIC sheaths and the accessories KIC connector and KIC sheath expander, allows ISC/IAC without detaching the sheath from the penis. This protects the skin and makes it unnecessary to change the sheath after every catheterisation.

Contraindications

KIC sheaths are not suitable in case of allergies/intolerance to latex or other materials in the sheath or adhesive/adhesive tape used; skin abrasions on the penis (e.g., wounds, eczema, pressure sores); acute skin diseases (e.g., mycosis, bacterial skin infections); pronounced phimosis.

Important information on hygiene and period of wear

When using continence products, the hygiene guidelines of the national health authorities must be observed. We recommend changing the sheath (the adhesive tape, bag) after 24 hours. The period of wear may also be less than 24 hours depending on the skin type.

Information

- Product and trade name can be found on the label.
- Available in the *Original* and *Extra thin* variants.
- KIC connector: change after three uses; rinse with water before connecting to a new sheath; removable.
- KIC sheath expander: for multiple uses; clean with water regularly.

Risks and warnings

Ask your doctor whether latex urinary sheaths are suitable for you for treating incontinence. Instructions on how to use urinary sheaths should be provided by a qualified nurse.

Can lead to skin irritation/changes. Allergic reaction to latex possible. Inspect the skin; if any abnormalities occur, stop using the product and seek medical advice.

If urinary sheaths are too tight or adhesives are used improperly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constrictions/necrosis. In this case, seek a doctor without delay. Note the information on selecting the size and the instructions for use of adhesives.

Information on selecting the size of the KIC urinary sheath

Selecting the correct size ensures a good fit and prevents leakage and pressure sores. To select the size, the circumference of the penis must be measured around the middle of the shaft. If it is adhered during an erection, the circumference is also meas-

ured on the erect penis. The product-specific measuring tapes state the diameter based on the circumference measurement.

Instructions for use

Hold the sheath with the thumb and index finger, raise the opening of the sheath over the plate of the connector, align the connector in the middle of the opening (Fig. 1).

Press on the connector sleeve and twist to lock it in place (Fig. 2). Pull on the sheath and connector to check the connection (Fig. 3).

Then apply the sheath with a suitable adhesive, observing the respective instructions for use (e.g. SAUER skin adhesive, SAUER adhesive tape) Loosen the connector to disconnect the sheath from the inlet tube of the urine collection bag (Fig. 4).

Guide the teeth of the sheath expander into the opening of the sheath (Fig. 5).

Widen the sheath by closing the scissor handles until they touch (Fig. 6).

Move the sheath down using the sheath expander until the glans is completely exposed (Fig. 7).

Disinfect and catheterise, following the instructions for use for the catheter used (Fig. 8).

Dry the connection sites on the connector and sheath to ensure a reliable connection. Then move the sheath forward again with the sheath expander. (Fig. 9)

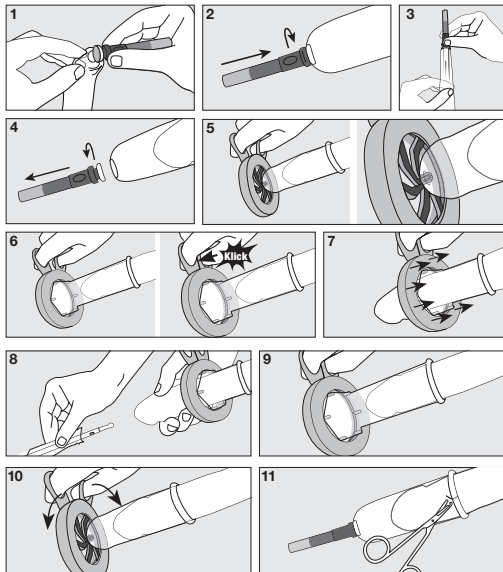
Close and detach the sheath expander (Fig. 10).

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 11). Do not use scissors with pointed tips as they may cause injury. The incision should be circular in shape so that the sheath does not tear. When cutting or tearing off, press down on the adhesive area of the sheath at the same time.

Reattach the connector, etc. (Fig.1,2,3,...).

Disposal

Dispose of the KIC sheath, connector and sheath expander in residual waste.

Connecting to the bag

Press the coupling tube of the KIC connector onto the bag adapter and press down gently all around. For bags without adapter, push the connecting tube directly on the inlet tube. Make sure that the buffer zone and inlet tube are not twisted or bent.

**Should you have any questions or problems, please turn to a medical professional or to our customer adviser:
MANFRED SAUER GmbH · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100**

If serious incidents occur when using the KIC system, they must be reported to the manufacturer and competent authority of the country in which the user resides.

Manfred Sauer GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines and the corresponding standards of care.

	Catalog number
	Store in a dry place
	Quantity
	Keep away from sunlight
	Swiss authorised representative
	UK authorised representative
	Batch number
	Observe instructions for use
	CE marked medical device
	Use by (year-month-day)
	Not for reuse
	Pharmaceutical central no.
	Drug Tariff
	List of products and services
	Manufacturer
	Latex

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