# SAUER-NephSys BEUTEL

STERIL

Deutsch	2
English	4
Français	6
Italiano	8
Nederlands	10
Svenska	12
Dansk	14
Suomi	16
Norsk	18
Eesti	20
Español	22
Português	24
Ελληνική	26
Magyar	28
Polski	30
Čeština	32
Română	34
Български	36
Slovenščina	38
Hrvatski	40
Srpski	42
Bosanski	44
Македонски	46
49	فارسی
51	العربية

## 2 Deutsch

### SAUER-NephSys Beutel (721.xxxx.SE), steril

#### Informationen

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### SAUER NephSys bag (721.xxxx.SE), sterile

#### Information

Instructions on how to use SAUER NephSys products should be provided by a medical expert. SAUER NephSys bags are for single use only. The SAUER NephSys belt set can be used multiple times. Please note laundering instructions on the label. The service life of the belt depends on individual usage (putting on, taking off, washing, etc.). In general, it is 2-3 months. Details of the device's trade name and performance characteristics can be found on the label.


#### Intended use and indications

The SAUER NephSys belt set (57.42, 57.43) is an accessory for the SAUER NephSys bag. The SAUER NephSys belt set is used to attach one or two SAUER NephSys bags to the hips. In combination, their intended use is to collect urine in persons with drainage systems above the reproductive organs, especially from nephrostomy catheters.

#### Risks and warnings


- The SAUER NephSys bag must a) not be reused and b) not be used if the sterile barrier is damaged, to prevent increasing the risk of infection.
- Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening urosepsis. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention.
- To reduce sources of infection, do not touch the connecting area of the bag adaptor or bring it into contact with surfaces (inner side of the Luer lock adapter, outside of stepped adaptor). To cut the inlet tube, use only scissors that have been disinfected beforehand (use a suitable disinfectant, e.g. for surfaces).
- Ensure that the bag is not attached above the level of the drainage system to allow urine to drain unobstructed (e.g. do not attach it above the connector of the nephrostomy catheter). Check for urine flow into the bag several times a day. Prevent the inlet tube from being twisted, kinked, or pinched.
- To prevent the unintentional opening/loosening of the belt set components, they must be replaced when they show signs of wear and tear.
- Avoid tension on the catheter. Note the following:
  - Connect the adapter of the bag with the catheter connector without tension.
  - Always separate the connection between the bag and the catheter before the waist belt is opened to prevent sudden tension on the catheter from the weight of the bag.
- When connecting the SAUER NephSys bag to a suitable bag for the night, ensure that the drain tap is opened and does not close during the night.

### SAUER NephSys bag stepped adaptor included, (push on), sterile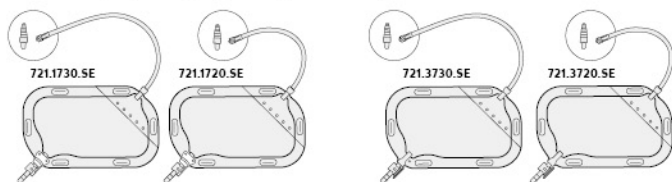

## Instructions for use


### 1. Applying and shortening the waist belt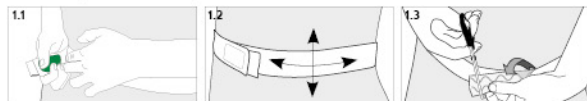

Place the side of the belt with the silicone lines on the skin and put the belt on anywhere between the waist and the hips under slight tension. Ensure a comfortable, but most importantly, a stable fit. The excess length of the belt can be cut off with scissors (1.3).

### 2. Positioning and fixating the hook-and-loop pad

Attach the hook-and-loop pad on the side of the body where the drainage system, e.g. the nephrostomy catheter, is located. Attach the hook-and-loop pad to the waist belt as in the illustration (Fig. 2.1, 2.a) in the middle between the side of the hip and the navel.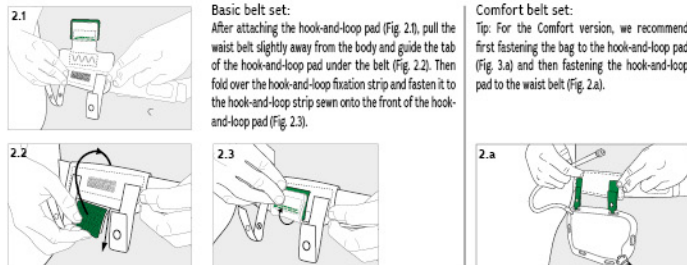


#### Basic belt set:

After attaching the hook-and-loop pad (Fig. 2.1), pull the waist belt slightly away from the body and guide the tab of the hook-and-loop pad under the belt (Fig. 2.2). Then fold over the hook-and-loop fixation strip and fasten it to the hook-and-loop strip sewn onto the front of the hook-and-loop pad (Fig. 2.3).


#### Comfort belt set:

Tip: For the Comfort version, we recommend first fastening the bag to the hook-and-loop pad (Fig. 3.a) and then fastening the hook-and-loop pad to the waist belt (Fig. 2.a).

### 3. Attaching the bag to the hook-and-loop pad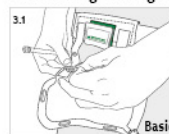

We recommend holding the inlet tube with one hand to prevent contact with the connecting area of the adapter. Attach the bag so that the outlet tap is facing the crotch. Thread the two retaining tabs of the hook-and-loop pad through the belt loops of the bag and close the snaps.

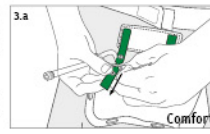
### 4. Connecting the bag to the catheter

Attach the male luer lock connector of the bag to the female luer lock connector of the ne-

phrostomy catheter by turning clockwise.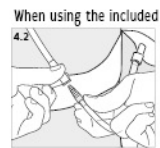

When using the included stepped adaptor, cut the inlet tube to the desired length (straight cut) with scissors that have been disinfected beforehand (use a suitable disinfectant, e.g. for surfaces). Push the adaptor onto the tube without touching the connecting area of the adaptor. Remove the protective cap from the adaptor. Connect the adaptor of the bag to the catheter connector without tension.


### 5. Securing the inlet tube with strain relief

To reduce the risk of twisting the inlet tube or putting strain on the catheter, the tube is secured to the waist belt in a suitable position using strain relief.



#### Basic belt set:

Removing the protective films from the adhesive fixation dot. Guide the tube between the adhesive surfaces of the adhesive fixation dot where the inlet tube and the waist belt cross (Fig. 5.1), press them together and fasten the adhesive dot to the waist belt. As a last step, fasten the adhesive fixation dot using the hook-and-loop fixation strip as in Fig. 5.2.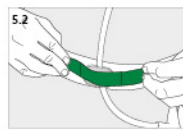

#### Comfort belt set:

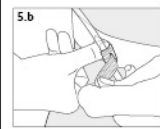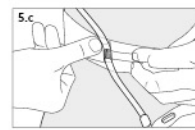Thread the fixation tab around the inlet tube as in Fig. 5.a. Press on the wide part of the hook-and-loop tab on the waist belt where the inlet tube and the waist belt cross (Fig. 5.b). Pull the hook-and-loop tab tight enough that the inlet tube can no longer be pulled out (do not pinch the tube). Then press the thin strip on the belt (Fig. 5.c).

### At night

The SAUER NephSys bag can remain fastened to the belt at night. However, it may be more comfortable to remove the bag from the belt and position it safely in bed next to the body. The SAUER NephSys bag can be connected to a suitable bag to accommodate an increased volume at night. For this, the connecting tube is slid onto the outlet tap of the SAUER NephSys bag and connected to the adaptor of the night bag. In this case, the outlet tap of the bag must remain open.

### Removal and changing

To empty the SAUER NephSys bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again.

To change the bag, carefully detach it from the catheter without touching the open ends of the connections.

### Disposal

Empty the SAUER NephSys bag before disposal and dispose of it in residual waste. Dispose of the belt system in household waste.

	Article number		Pharmaceutical central no.
	Store in a dry place		Drug Tariff
	Do not resterilise		List of products and services
	Quantity		Do not use if package is damaged
	Keep away from sunlight		Date of manufacture
	Swiss authorised representative		Manufacturer
	Sterilised with ethylene oxide	If serious incidents occur when using SAUER urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.	
	Single sterile barrier system		
	Batch number	MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.	
	Observe instructions for use		
	CE marked medical device		
	Use by (year-month-day)		
	Not for reuse	Neurott 7-15 · 74931 Lobbach · Germany Tel. +49 6226 960 104 · export@manfred-sauer.de www.manfred-sauer.com	

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