97.xx	97.7501.xx	97.6000.xx	97.6001.xx	97.4511.xx
-------	------------	------------	------------	------------

Selbstklebendes Kondom-Urinal	Deutsch	2
Self-Adhesive Urinary Sheath	English	4
Étui pénien autocollant	Français	6
Condom urinario autoadesivo	Italiano	8
Zelfklevende externe katheters	Nederlands	10
Självhäftande urindroppssamlare	Svenska	12
Selvklebende urinalkondom	Dansk	14
Itseliimautuva urinaalikondomi	Suomi	16
Selvimende uridom	Norsk	18
Sjálflímandi þvagsmokkar	Ísland	20
Isekleepuv uriinikogur	Eesti	22
Colector de orina peneano autoadhesivo	Español	24
Preservativo urinário autoadesivo	Português	26
Αυτοκόλλητος περιπεϊκός καθετήρας	Ελληνική	28
Öntapadó vizeletkondom	Magyar	30
Samoprzylepny cewnik zewnętrzny	Polski	32
Samolepicí urinální kondom	Čeština	34
Prezervativ urinar autoadeziv	Română	36
Самозалепващ уринален кондом	Български	38
Samolepilni urinal kondom	Slovenščina	40
Samoljepivi urinarni kondom	Hrvatski	42
Samolepljivi urinarni kondom	Srpski	44
Samoljepljivi urinarni kondom	Bosanski	46
Самолеплив уринарен кондом	Македонски	48

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Self-Adhesive Urinary Sheath – SAUER Comfort


Ready for use, for single use

Contents

30 urinary sheaths, self-adhesive, made of synthetic material (without addition of latex), powdered, manufactured as one piece with the connecting tube, connects to all leg bags and bed bags with and without adapter

Order no.	Colour	length of sheath	buffer zone 	adhesive zone 	adhesive
97.xx	white	130 mm	75 mm	50 mm	normal
97.7501.xx	blue	130 mm	75 mm	50 mm	strong
97.6000.xx	green	115 mm	60 mm	50 mm	normal
97.6001.xx	red	115 mm	60 mm	50 mm	strong
97.4511.xx	violet	75 mm	45 mm	25 mm	strong

- 11 sizes:** Ø 18, 20, 22, 24, 26, 28, 30, 32, 35, 37, 40 mm
("xx" in the order no. represents the size of the sheath, e.g. 97.30 = 30 mm)
- 1 protective cloth**, matching the size of the sheath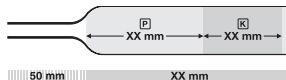

Intended use

SAUER Comfort urinary sheaths are used for external urinary drainage into a urine collection bag in men with urinary incontinence or if external urinary drainage is desired for other reasons. They are used on intact skin.

Indications

Urinary incontinence of varying causes without residual urine (e.g., neurogenic urinary dysfunctions, hyperactive bladder, urinary incontinence after surgery); urinary drainage in people who cannot manage to go to the toilet by themselves.

Contraindications

Chronic urinary retention with high intravesical pressure; sphincter dyssynergia with significant urinary retention; constrictions in the urethra (e.g., prostatic hyperplasia) resulting in high residual amounts of urine; skin abrasions on the penis (e.g., wounds, eczema, pressure sores); acute skin diseases (e.g., mycosis, bacterial skin infections); allergies/incompatibilities to the material or adhesive/adhesive spray/adhesive tape; pronounced phimosis.

Important information on hygiene and period of wear

When using continence products, the hygiene guidelines of the national health authorities must be observed. We recommend changing the sheath (the adhesive tape, bag) after 24 hours. The period of wear may also be less than 24 hours depending on the skin type.

Selecting the right size/product

Sauer Comfort Urinary Sheaths are available in 5 versions, each with up to 11 sizes. It is essential to select the right version and

size so that the sheath fits properly and does not cause leaks or pressure sores. To select the size, the circumference of the penis must be measured around the middle of the shaft. If the sheath is adhered during erection (see application), the circumference is also measured on the erect penis. The product-specific measuring tapes state the diameter based on the circumference measurement. If leaks or pressure sores occur, the sheath size must be checked and adjusted if necessary.

Risks and warnings

Ask your doctor if urinary sheaths are suitable for your incontinence care. Instructions on how to use urinary sheaths should be provided by a qualified nurse.

The use of urinary sheaths may cause skin irritation and skin changes. When removing a urinary sheath, inspect the skin. If you notice any abnormalities, stop using the product and seek medical advice.

If urinary sheaths are worn too tightly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constrictions and necrosis. In this case, seek a doctor without delay.

APPLICATION

Preparations

Wash the skin in the whole genital area thoroughly and dry well. If used on damp skin or on skin that was pre-treated with oily/greasy products, the adhesive effect is reduced.


An erection is not absolutely necessary for adhesion, but if one can be manipulated, we recommend that you apply the sheath to

an erect penis as this gives a more stable surface for adhesion and prevents constriction.

Optional: Pull over the protective cloth so that the pubic hair does not stick or obstruct adhesion.

Applying the sheath

Unroll the buffer zone. Depending on the version of the sheath, unroll it 1–3 folds. For example, 1 fold for the violet sheaths, 2 folds for the green and red sheaths, and 3 folds for the white and blue sheaths (Fig. 1). This unrolled area provides the required buffer zone between the penis and the tip of the sheath, as well as the correct position of the adhesive zone. If the sheath is difficult to unroll, it helps to stretch the connecting tube slightly.

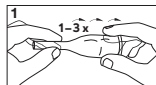
Unroll 1–3 folds

Apply the sheath

Unroll the sheath further

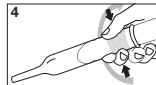Press the sheath around and against the penis

Remove the ring

Position the sheath on the foreskin without smoothing the foreskin back (Fig. 2). If an erection is not possible, pull the foreskin gently to smooth the skin. If there is no foreskin, place the urinary sheaths directly on the glans.

When unrolling further, you reach the adhesive zone (Fig. 3). To make unrolling easier, you can also stretch the connecting tube here. From the start, unroll the sheath evenly all around.

Avoid any folds in the adhesive area. Check the position of the sheath in the adhesive area and press it carefully around and against the shaft (Fig. 4). If necessary, rub out longitudinal folds and air bubbles.

Removing the ring

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 5). Do not use scissors with pointed tips as they may cause injury.

Removing the ring

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 5). Do not use scissors with pointed tips as they may cause injury.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7–15 · 74931 Lobbach · Germany · +49 6226 960100

If the sheath is unrolled unevenly towards one side, it may stick and then be almost impossible to unroll.

The incision should be circular in shape so that the sheath does not tear. When cutting or tearing off, press down on the adhesive area of the sheath at the same time.

Connecting to the bag

Push the connecting tube of the sheath on the adapter of the bag and press all around gently. For bags without adapter, push the connecting tube directly on the inlet tube. Make sure that the buffer zone and inlet tube are not twisted or bent.

Removing the sheath

Remove the sheath after 24 hours at the latest and pull or roll off from the back towards the tip of the penis. Do not pull too strongly so as to avoid skin abrasions. An oil pad can be used to undo the adhesive bond. Carefully remove any adhesive remains.

Disposal

Dispose of the sheath in household waste.

	Catalog number		Observe instructions for use
	Store in a dry place		CE marked medical device
	Quantity		Use by (year-month-day)
	Keep away from sunlight		Not for reuse
	Swiss authorised representative		Pharmaceutical central no.
	UK authorised representative		Drug Tariff
	Batch number		List of products and services
			Manufacturer

If serious incidents occur when using "SAUER Comfort" urinary sheaths, these must be reported to the manufacturer and competent authority of the country in which the user resides.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7–15 · 74931 Lobbach · Germany · +49 6226 960100

Manfred Sauer UK Ltd · Barn Way · Northampton NN5 7UW
United Kingdom · Phone +44 1604 595696

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