Gebrauchsanweisung	Deutsch	2
Instructions for use	English	4
Notice d'utilisation	Français	6
Istruzioni per l'uso	Italiano	8
Gebruiksaanwijzing	Nederlands	10
Bruksanvisning	Svenska	12
Brugsanvisning	Dansk	14
Käyttöohje	Suomi	16
Bruksanvisning	Norsk	18
Kasutusjuhised	Eesti	20
Instrucciones de uso	Español	22
Instruções de utilização	Português	24
Οδηγίες χρήστης	Ελληνική	26
Használati utasítás	Magyar	28
Instrukcja użytkowania	Polski	30
Návod k použití	Čeština	32
Instrucțiuni de utilizare	Română	34
Инструкция за употреба	Български	36
Navodila za uporabo	Slovenščina	38
Upute za uporabu	Hrvatski	40
Upustvo za upotrebu	Српски/Srpski	42
Upustvo za upotrebu	Bosanski	44
Упатство за употреба	Македонски	46
دستور العمل استفاده	فارسی	48
قسطرة التبؤل الخارجي ذاتية الالتصاق	العربية	50

Deutsch

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www.manfred-sauer.com

English

Single use gel catheter for ISC (Intermittent self-catheterisation)

- Ready for use
- The catheter with 2 functional zones (rounded tip and guiding zone)
- Functional packaging for variable use
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 4 sizes: CH 10, CH 12, CH 14, CH 16

Group of Users: IQ3104.xx, IQ3114.xx – Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:

The first use should be supervised by a professional.

If liquid has already leaked from the gel sachet into the packaging, the catheter may no longer be used.

With gel catheters, there may sometimes be a slight delay after insertion before the urine begins to flow.

Risks and Cautions:

The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.


Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel as it is not lubricated. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15–25°C.

Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.


- | | |
|---|------------------------------|
| Article number | REF |
| Serial number | SN |
| Store in a dry place | umbrella |
| This way up | double arrows |
| Do not resterilise | crossed-out sterilize |
| Quantity | film strip |
| Keep away from sunlight | sun |
| Sterilised with ethylene oxide | empty oval |
| Single sterile barrier system | single barrier system |
| Single sterile barrier system with protective packaging outside | protective packaging outside |
| Batch number | lot |
| Observe instructions for use | flag |


- | | |
|---------------------------------|----------------------------------|
| CE ^{MD} 0123 | CE marked medical device |
| square with a cross (X) | Use by (year-month-day) |
| circle with a diagonal line (X) | Not for reuse |
| PZN | Pharmaceutical central no. |
| DT | Drug Tariff |
| LPP | List of products and services |
| cube with a diagonal line (X) | Do not use if package is damaged |
| wavy line (Date of manufacture) | Date of manufacture |
| wavy line (Manufacturer) | Manufacturer |
| CH REP | Swiss authorised representative |
| globe icon | Importer |

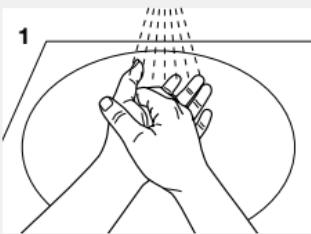
Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly. You can use the functional packaging as gripping aid.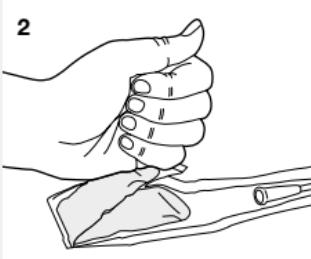


Open the gel sachet by folding and squeezing. Make sure the whole content of the sachet is emptied into the packaging.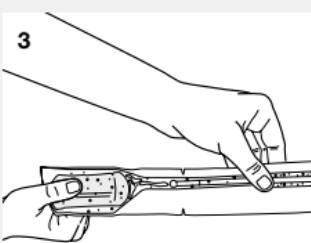


Spread the sterile lubricating gel along the full length of the catheter (except for the funnel) to make sure all of the surface is coated with gel.
Use tear marks at the front end to open the functional packaging and then tear off with a jerk including the gel sachet.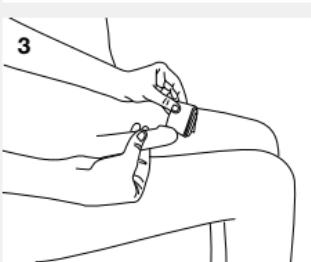


Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).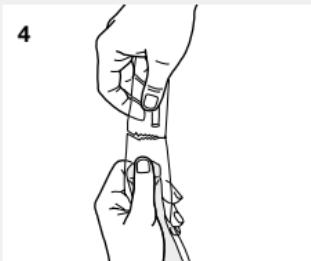

Open the packaging at the back end.
If the catheters sticks to the packaging please pull out twisting it slowly at the same time.
Attach a urine bag to the catheter if necessary.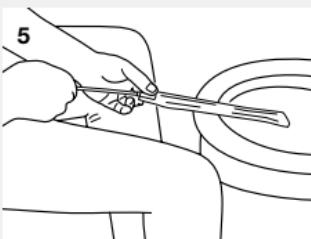

The packaging can be used as an extension for draining the urine directly into the toilet. Move the funnel towards the front to seal the packet.

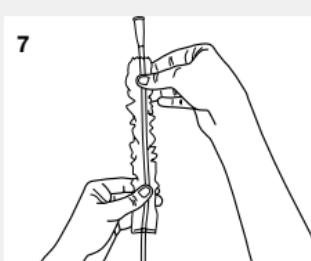Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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