IQ4304.xx

IQ4303.xx

***IQ*CATH**

Gebrauchsanweisung	Deutsch	2
Instructions for use	English	4
Notice d'utilisation	Français	6
Istruzioni per l'uso	Italiano	8
Gebruiksaanwijzing	Nederlands	10
Bruksanvisning	Svenska	12
Brugsanvisning	Dansk	14
Käyttöohje	Suomi	16
Bruksanvisning	Norsk	18
Kasutusjuhised	Eesti	20
Instrucciones de uso	Español	22
Instruções de utilização	Português	24
Οδηγίες χρήσης	Ελληνική	26
Használati utasítás	Magyar	28
Instrukcja użytkowania	Polski	30
Návod k použití	Čeština	32
Instrucțiuni de utilizare	Română	34
Инструкция за употреба	Български	36
Navodila za uporabo	Slovenščina	38
Upute za uporabu	Hrvatski	40
Uputstvo za upotrebu	Српски/srpski	42
Uputstvo za upotrebu	Bosanski	44
Упатство за употреба	Македонски	46
دستور العمل استفاده	فارسی	48
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	50

Deutsch

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English

Hydrophilic Single-use catheter for ISC (Intermittent self-catheterisation)

- Immediately ready for use
- Catheter with 2 functional zones with sleeve (rounded tip and guiding zone)
- Peel packaging for easy handling
- Sleeve for No-touch insertion

IQ4303.xx – Boys Total length: ca. 330 mm, length without funnel: ca. 300 mm
3 sizes: CH 10, CH 12, CH 14

IQ4304.xx – Men Total length: ca. 430 mm, length without funnel: ca. 400 mm
5 sizes: CH 10, CH 12, CH 14, CH 16, CH 18

Group of Users: IQ4303.xx – Boys, IQ4304.xx – Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:

The first use should be supervised by a professional.

If the position of the sleeve and/or the clip is changed when the catheter is taken out of the "Peel" packaging, the catheter may not be used (see Fig. 7).

To have your hands free during disinfection, the catheter can be attached to a suitable surface with the adhesive dot.

Risks and Cautions:

The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.

Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use if the sterile packaging or the catheter is damaged, as there is a risk of infection. We recommend an operating temperature range of 15-25°C.

Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

Catalog number

Serial number

Store in a dry place

This way up

Do not resterilise

Quantity

Keep away from sunlight

Sterilised with ethylene oxide

Single sterile barrier system

Single sterile barrier system with protective packaging outside

Batch number

Observe instructions for use

CE marked medical device

Use by (year-month-day)

Not for reuse

Pharmaceutical central no.

Drug Tariff

List of products and services

Do not use if package is damaged

Date of manufacture

Manufacturer

Swiss authorised representative

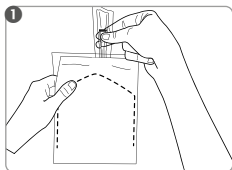
Rounded tip


Immediately ready for use


Important! Aluminium pouches containing 10 catheters each provide an additional layer of protection against drying up. Condensation may form in these pouches occasionally, which does not affect the quality of the product. Use the catheters within 30 days of opening the aluminium pouch.

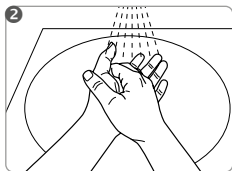
Open the aluminium bag and take one catheter.

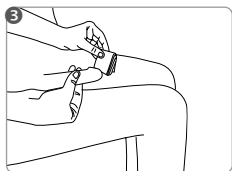
Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly.

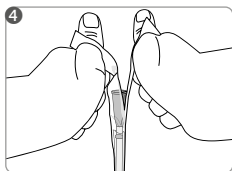
Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

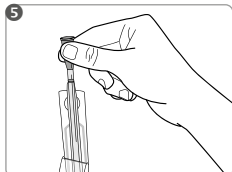
Hold the catheter vertically with the tip downwards and open the "Peel" packaging at the funnel end.

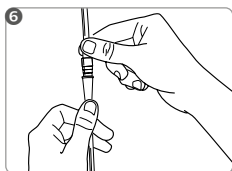Remove the catheter from the plastic peel film holding it at the funnel. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

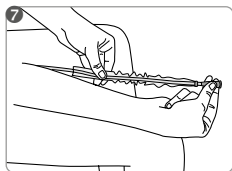If necessary, attach a urine bag to the catheter.

If the position of the sleeve and/or the clip is changed when the catheter is taken out of the "Peel" packaging, the catheter may not be used. Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly

Dispose of the catheter according to local authority guidelines.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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