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If serious incidents occur when using double-sided adhesive tape, they must be reported to the manufacturer and competent authority of the country in which the user resides.

The MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to the instructions for use or from neglecting the hygiene guidelines.

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Informationen und ingredients

- Für single use only.
- The trade name of the product can be found on the label.
- Available in the variants *Extra film* (made of hydrokolloid) and *Foam Strip* (made of polyethylene, acrylate copolymer adhesive).

Risks and warnings

- If adhesive tape is applied too tightly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constriction/necrosis. In this case, seek a doctor without delay. If possible, apply during an erection and do not stretch the adhesive tape while applying to prevent constriction.
- Due to material rashes and overlapping the edges of the bladder, the diameter is increased by the thickness of the material and overlapping tapes. To prevent pressure sores/constrictions, try to use the next larger sheath.
- Skin changes/irritation may occur. Check the skin carefully, especially if any abnormalities occur, stop using the product and seek medical advice.

Directions
SAUER urinary sheath attachment procedure

1 Apply to dry skin that has not been pre-treated. Cover any pubic hair with a cloth. Apply the adhesive tape to the penis in an overlapping spiral not too tightly (Fig. 1). Do not stretch the tape. Then unroll the urinary sheath as far as desired and position the sheath on the tip of the penis (without retracting the foreskin. If present, slightly stretch the connecting tube. Use one hand to unroll the sheath over the adhesive band on the shaft, while using the other hand to keep the penis taut. Press the sheath onto the penis. Retract the sheath gently grasp the adhesive site with your hand for about 60 seconds. The body heat develops the full adhesive strength. The ring remaining beyond the adhesive area can be cut open or cut/turn off completely using scissors with rounded tips. Do not use any object that could apply pressure sores or if the penis retracts considerably (Fig. 2).

2 Remove the urinary sheath. For hygiene reasons, the sheath should be removed after 24 hours at the latest by pulling it off carefully towards the tip of the penis to prevent skin damage.

Dispose of the sheath and adhesive in household waste.

Removal of SAUER urinary sheath

For hygiene reasons, the sheath should be removed after 24 hours at the latest by pulling it off carefully towards the tip of the penis to prevent skin damage.

Dispose of the sheath and adhesive in household waste.

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