|                                     |               |    |
|-------------------------------------|---------------|----|
| Gebrauchsanweisung                  | Deutsch       | 2  |
| Instructions for use                | English       | 4  |
| Notice d'utilisation                | Français      | 6  |
| Istruzioni per l'uso                | Italiano      | 8  |
| Gebruiksaanwijzing                  | Nederlands    | 10 |
| Bruksanvisning                      | Svenska       | 12 |
| Brugsanvisning                      | Dansk         | 14 |
| Käyttöohje                          | Suomi         | 16 |
| Bruksanvisning                      | Norsk         | 18 |
| Kasutusjuhised                      | Eesti         | 20 |
| Instrucciones de uso                | Español       | 22 |
| Instruções de utilização            | Português     | 24 |
| Οδηγίες χρήσης                      | Ελληνική      | 26 |
| Használati utasítás                 | Magyar        | 28 |
| Instrukcja użytkowania              | Polski        | 30 |
| Návod k použití                     | Čeština       | 32 |
| Instrucțiuni de utilizare           | Română        | 34 |
| Инструкция за употреба              | Български     | 36 |
| Navodila za uporabo                 | Slovenščina   | 38 |
| Upute za uporabu                    | Hrvatski      | 40 |
| Uputstvo za upotrebu                | Српски/srpski | 42 |
| Uputstvo za upotrebu                | Bosanski      | 44 |
| Упатство за употреба                | Македонски    | 46 |
| دستور العمل استفاده                 | فارسی         | 48 |
| قسطرة التبول الخارجي ذاتية الالتصاق | العربية       | 50 |

# Deutsch

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## English

### Hydrophilic Single-use catheter for ISC (Intermittent self-catheterisation)

- Not ready for use
- The catheter with 2 functional zones (rounded tip and guiding zone)
- Peel packaging for easy handling
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 5 sizes: CH 10, CH 12, CH 14, CH 16, CH 18

Group of Users: IQ2904.xx – Men

#### Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

#### Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

#### Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

#### Instructions for use:

The first use should be supervised by a professional.


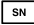
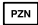
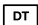
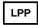
To have your hands free while activating the catheter and during disinfection, the catheter can be attached to a suitable surface with the adhesive dot.


#### Risks and Cautions:

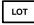The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.


Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without activating with sterile saline solution 0.9% first as it is not lubricated. Tap water affects catheter sterility and thus poses an increased infection risk. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C.

Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.


|  |   |   |                                  |
|--|---|---|----------------------------------|
|  | Article number  |  | CE marked medical device         |
|  | Serial number   |  | Use by (year-month-day)          |
|  | Store in a dry place  |  | Not for reuse                    |
|  | This way up   |  | Pharmaceutical central no.       |
|  | Do not resterilise  |  | Drug Tariff                      |
|  | Quantity  |  | List of products and services    |
|  | Keep away from sunlight   |  | Do not use if package is damaged |
|  | Sterilised with ethylene oxide                                  |  | Date of manufacture              |
|  | Single sterile barrier system                                   |  | Manufacturer                     |
|  | Single sterile barrier system with protective packaging outside |  | Swiss authorised representative  |
|  | Batch number  |  | Importer                         |
|  | Observe instructions for use                                    |   |                                  |

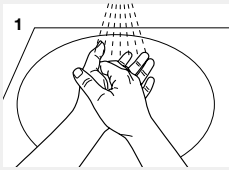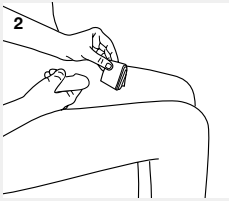
Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly.

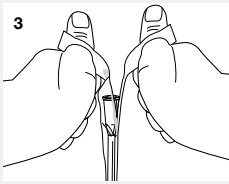
Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

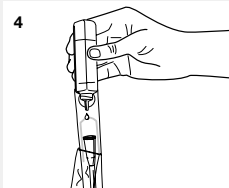
Hold the catheter vertically with the tip downwards and open the "Peel" packaging at the funnel end.

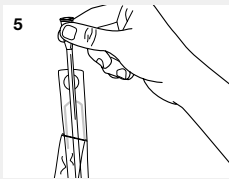
Fill in a sufficient quantity of sterile saline solution 0.9% to ensure that all of the catheter surface (with the exception of the funnel) is moistened for at least 30 sec.

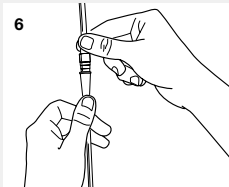Remove the catheter from the plastic peel film holding it at the funnel.  
If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

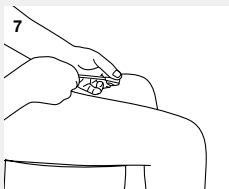If necessary, attach a urine bag to the catheter.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Dispose of the catheter according to local authority guidelines.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

**Should you have any questions or problems, please turn to a medical professional or to our customer adviser.**

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